**EN** Insert four AA size alkaline batteries in the battery compartment in the back of the AED Trainer 3.

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**EN** Press and hold both buttons for 6 seconds. Release both buttons, then press the lightning bolt button to scroll through list of languages until the desired language is announced. To save the language selection, turn off the AED Trainer 3 by pressing the power button. The language will be active the next time you turn on the AED Trainer 3.

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**EN** Apply "TRAINING ONLY" sticker for the selected language.

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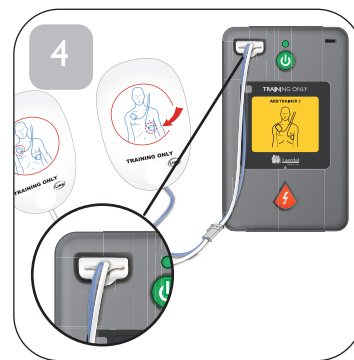
**EN** Plug the Training Pads connector into the connector port on the front of the AED Trainer 3. Prepare the manikin to accept pad placement as appropriate. You are now ready to start training.

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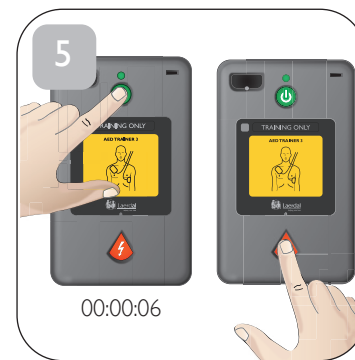
**EN** To select a different scenario, press and hold the power button for 6 seconds. Press the lightning bolt button as directed to scroll through the scenarios until the desired scenario is announced. To save the selected scenario, turn off the AED Trainer 3 by pressing the power button. The scenario will be active the next time you turn on the AED Trainer 3.

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
## Scenario codes
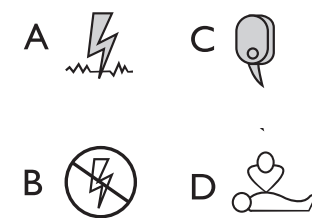
## EN Scenario codes

- A = Shockable Rhythm
- B = Non-shockable Rhythm
- C = Pads problem detected
- D = CPR First protocol initiated

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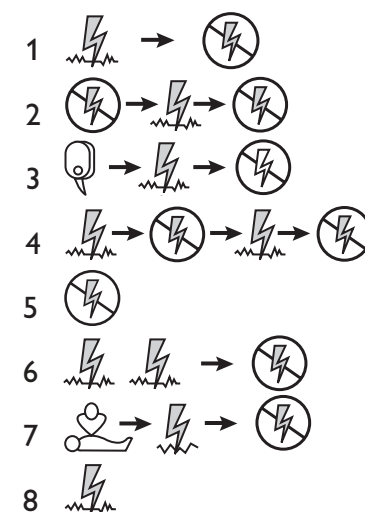
Distributed in the US  
by Laerdal Medical  
Corporation Wappingers Falls,  
NY 12590-8840

Distributed in Canada  
by Laerdal Medical  
Canada Ltd. Toronto,  
Ontario M1V 4C3

Made in China

© LAERDAL MEDICAL AS 2020. All rights reserved.  
Laerdal 20-03295 Rev D

# AED Trainer 3

## Quick Setup Guide

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