

VOYAGER AMBI®

USER GUIDE



IMPORTANT INFORMATION

1. Your Voyager Ambi® should only be used by trained operators.
2. Training on your Voyager Ambi® should be conducted in line with Paraid® guidelines, as stated in this user guide.
3. Training should be undertaken every 3 years, to ensure competency using the Voyager Ambi®.
4. Ensure training records are maintained and up to date (see training record on page 4).
5. Quarterly practice with your Voyager Ambi® chair is recommended, to ensure staff remain trained and competent.
6. Quarterly inspection of your Voyager Ambi® is recommended, to ensure the integrity of the chair is maintained. See page 24 for information.
7. We recommend two-person operation for heavier payload passengers.
8. As a Class 1 Medical Device, Paraid® offer service & maintenance of your Voyager Ambi® that should be undertaken by an approved service provider annually, to ensure safe operation. This includes a physical, as well as a visual check and replacement of worn parts.
9. Only genuine Paraid® parts should be used when maintaining your Voyager Ambi®.
10. To ensure the optimal performance of your Voyager Ambi®, belts should be rotated every 3 months.

Note: Whilst the products are designed to work in most stair applications, Paraid® accepts no liability for damage to stairs caused when the transfer chair is in use.

WARRANTIES & LIABILITIES

The Voyager Ambi® is guaranteed for five years against defective material and workmanship. This does not include parts subject to wear, such as 'V' belts, or damage caused by abuse or sabotage, and/or any other damage which has occurred due to a breaching of the guidelines contented in the User Guide. This includes getting your Voyager Ambi® serviced by a 3rd party company. 'V' belts are advised to be changed every 2 years on a basis of average use and practice.

Neither the manufacturer nor their agents shall make recompenses for repairs carried out by third parties or be liable for any damage, either incidental or consequential to property or persons. If you are dissatisfied with the product or service provided by Paraid®, please contact our customer service team on +44 (0)121 700 7455 or enquiries@paraid.com.

CONTENTS

4-5 Training & Maintenance Records

6-8 General Information

9-10 Deployment

11-12 Transfer

13-16 Descent

17-18 Lifting

19-20 Special Features

21 Training

22 Service & Maintenance

23 Wall Hook Installation

24 Visual Inspection

25 ProMove



TRAINING RECORD

Trainee Name	Training Date	Expiry Date	Trained by

MAINTENANCE RECORD

Date	Service or Inspection	Belts Changed	By	Company

NOTE: 'V' belts should be rotated every 3 months

VOYAGER AMBI®

The Voyager Ambi® is a narrow transfer chair, designed to safely assist in evacuating passengers, if a health emergency occurs, from an aircraft, boat or train. The Voyager Ambi® can descend stairs and be lifted over obstructions. Its front and rear grab handles enable up to four-persons to assist in a transfer. With a 150kg (23.6 stones) payload capacity and a 405mm width framework, it is ideal for navigating the narrowest aisles, stairs and walkways.



SAFE WORKING LOAD



FOR DESCENT AND LIFTING




Two-four person operation

HORIZONTAL OPERATION



NOT SUITABLE FOR SPIRAL STAIRCASES

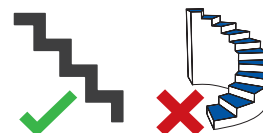

ADVISORY



Four-person operation is advised for heavier passengers

MAINTENANCE (STRONGLY ADVISED)



TRAINING (STRONGLY ADVISED)




MANUFACTURERS WARRANTY



STAIR ANGLE



MINIMUM ALLOWABLE STAIR WIDTH

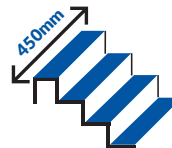

WHAT IS INCLUDED?


VOYAGER AMBI®



DUST COVER







WALL MOUNTED HOOKS


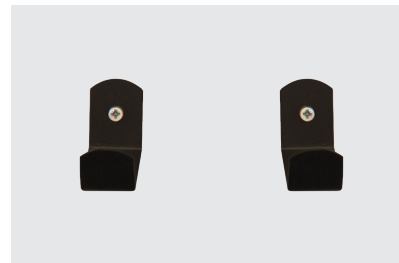



USER GUIDE




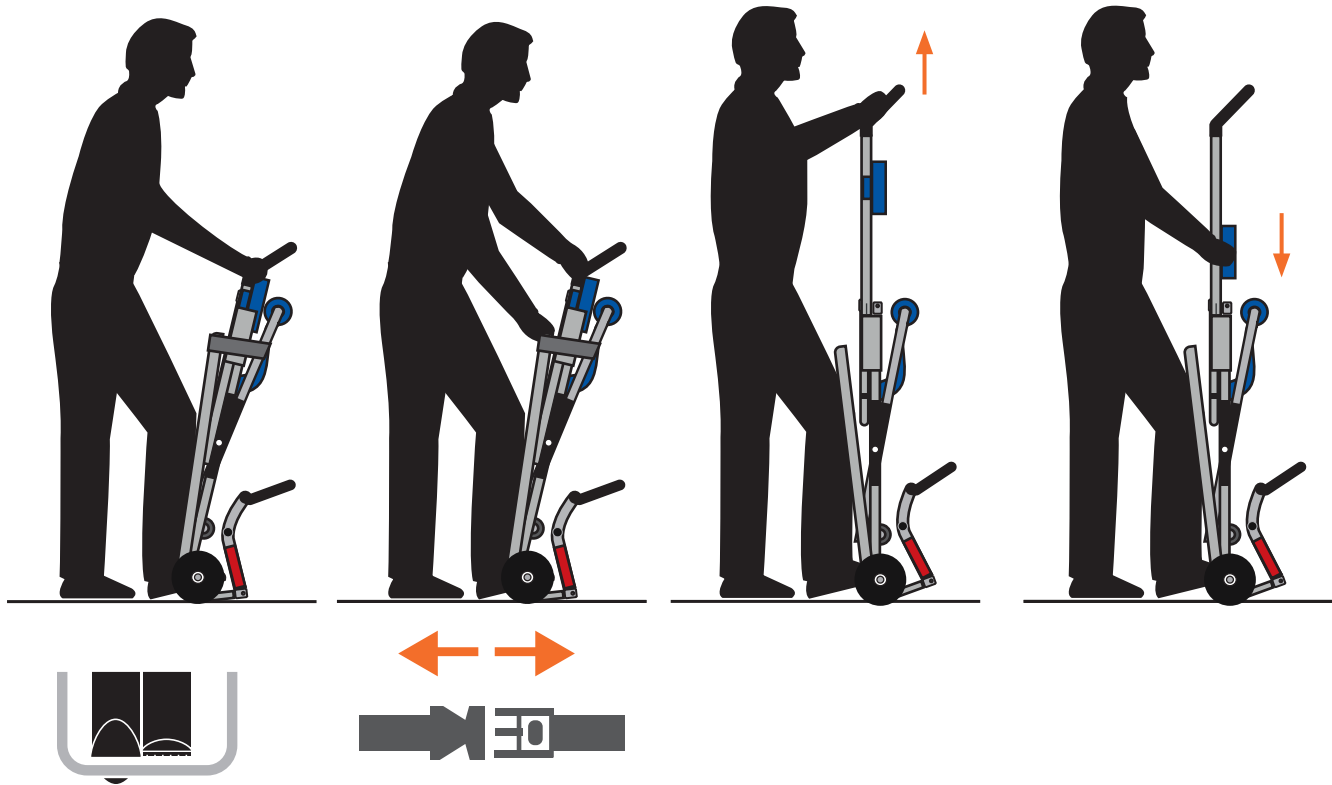


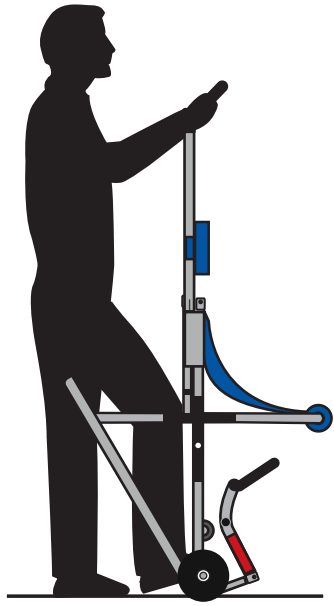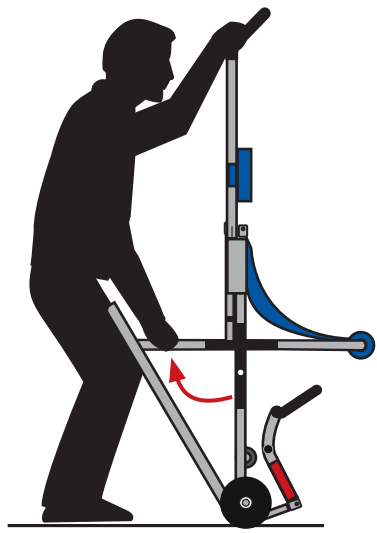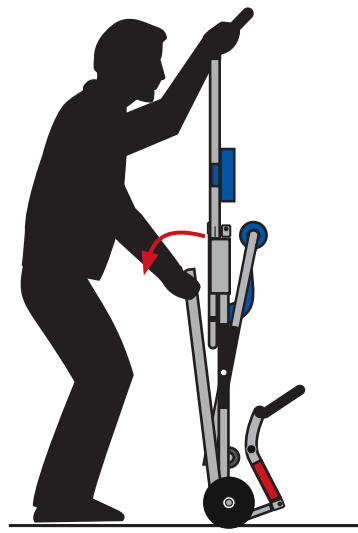


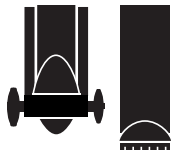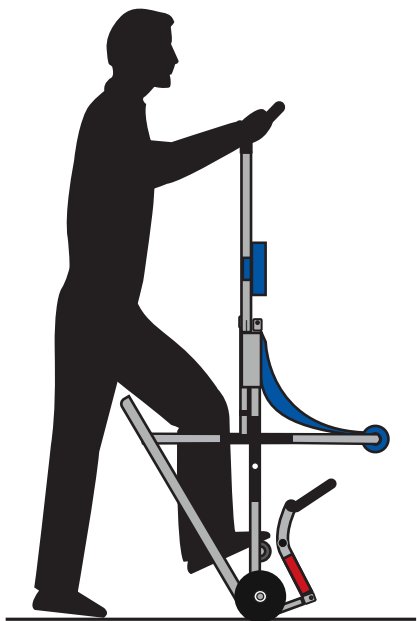

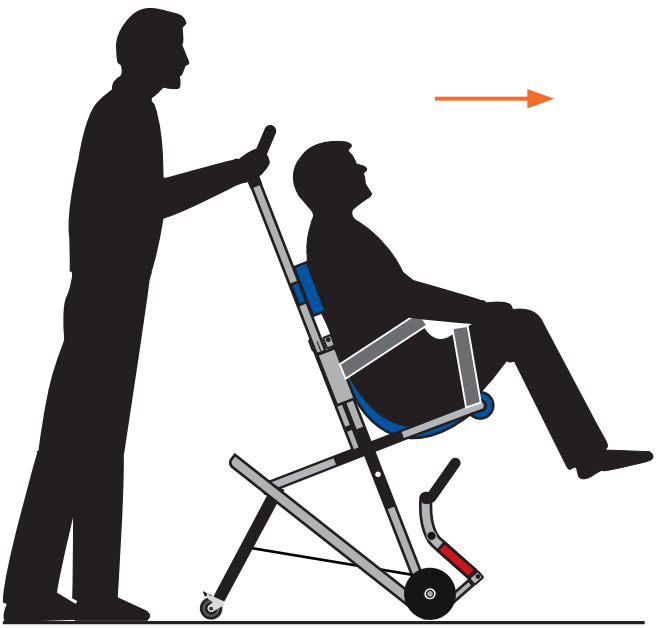

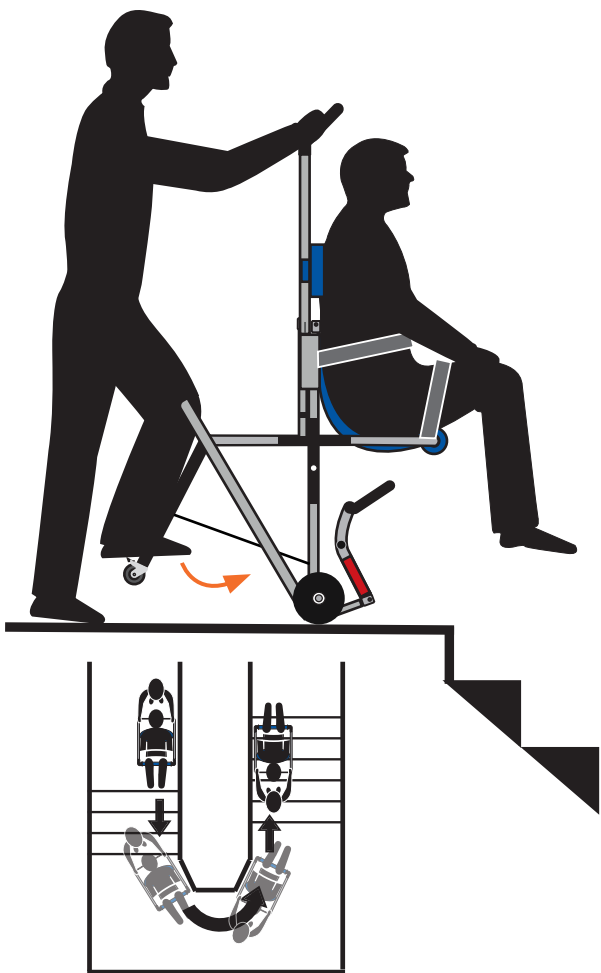

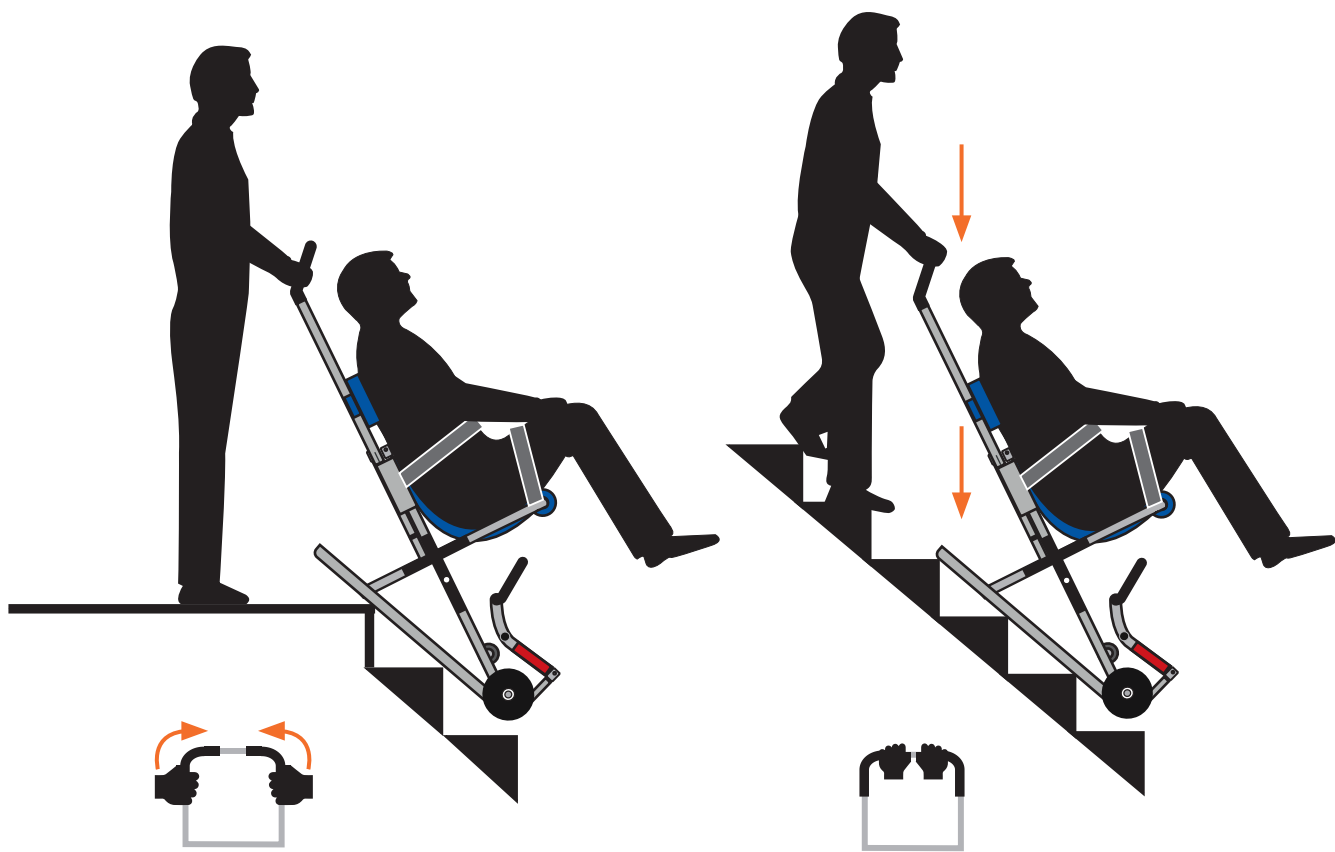


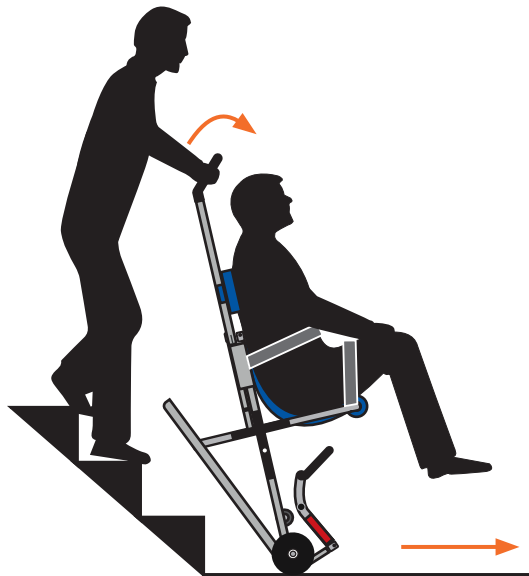







FRONT LIFTING HANDLE

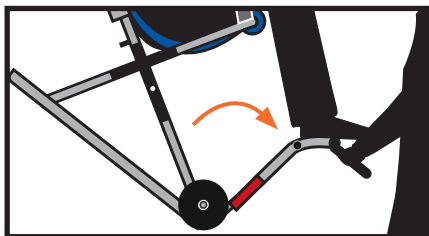


REAR LIFTING HANDLE

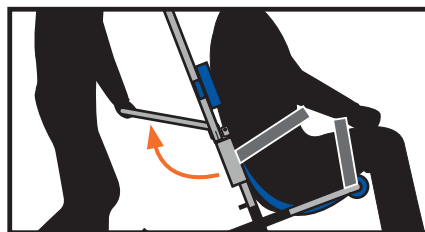

LIFTING

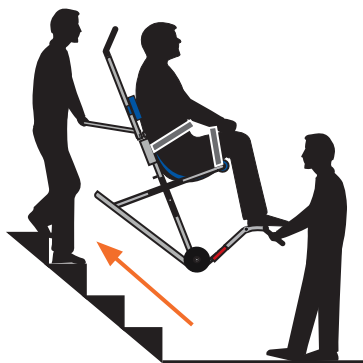


4 PERSON OPERATION

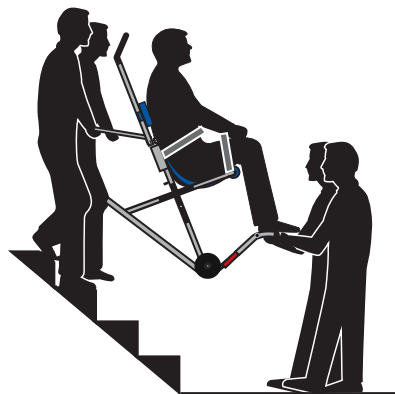


REAR BRAKING CASTORS




Unlocked

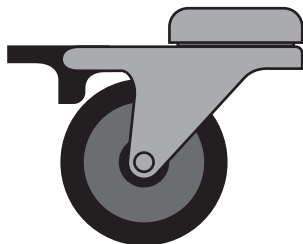

Locked

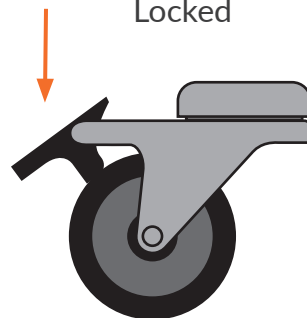

HEADREST

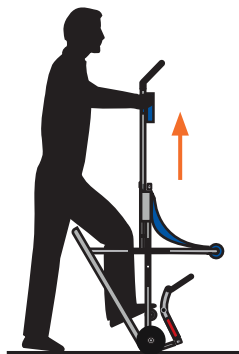

FOOT REST APPLICATION



Passenger should not place all their weight on foot rest




TRAINING

DO YOU HAVE TRAINED OPERATORS?

REASONS TO BE TRAINED:

- ✓ **IMPROVED SAFETY**
- ✓ **REDUCED RISK OF INJURY**
- ✓ **FASTER TRANSFER TIMES**
- ✓ **INCREASED EMPLOYEE CONFIDENCE**
- ✓ **PEACE OF MIND FOR EMPLOYERS**

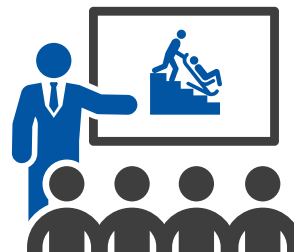

Bespoke Paraid Trainer Courses

The Paraid® training courses are designed to train delegates how to use Voyager Ambi® safely and correctly. Providing practical skills our training courses will ensure that your operatives are competent and confident in the use of Voyager Ambi®. Our bespoke courses will be tailored to your bespoke needs and requirements.

On successful completion, each delegate will be certified for three years.

To discuss your training requirements,
please call **+44 (0) 121 700 7455** or email
enquiries@paraid.com

SERVICE & MAINTENANCE

HAVE YOU ARRANGED A SERVICE AND MAINTENANCE CONTRACT?



As a Class 1 Medical Device, your Voyager Ambi® should be regularly serviced and maintained to ensure its safe operation. This requirement is included within the PUWER Regulation (Provision and Use of Works Equipment Regulations 1998 - UK only).

Our maintenance contracts provide you with complete peace of mind that you are covered and should an emergency situation occur, your transfer chair will be fully operational. Our contracts include labour and genuine replacement parts, ensuring your Voyager Ambi® is kept fully operational and meets its legal requirements.




To discuss your training requirements, please call **+44 (0) 121 700 7455** or email **enquiries@paraid.com**

WALL HOOKS - FOR WALL MOUNTING

INSTALLATION

Paraid® can install your transfer chair as well as provide expert advice and guidance on the most effective locations.

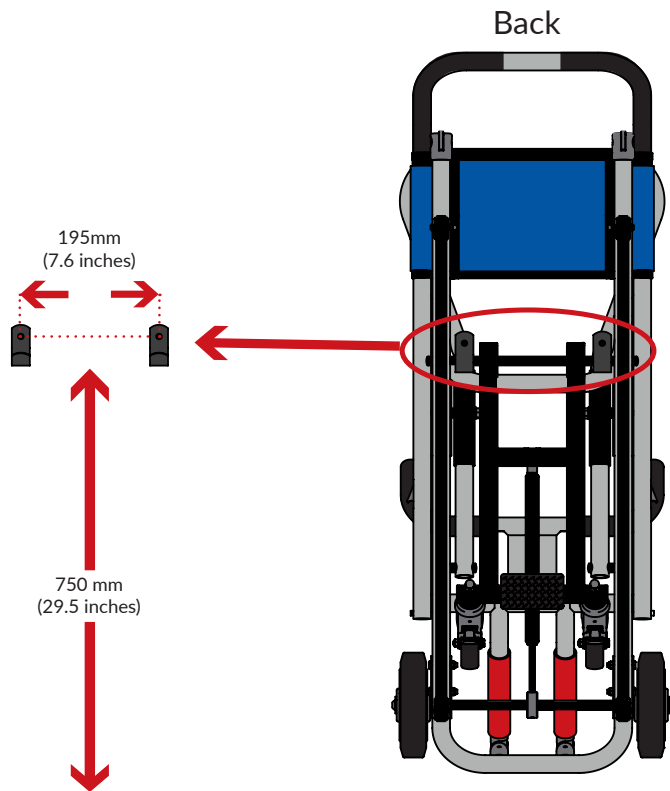

VISUAL INSPECTION

WE RECOMMEND YOU CARRY OUT A QUARTERLY INSPECTION TO ENSURE THE INTEGRITY OF THE VOYAGER AMBI® IS MAINTAINED.

DEPLOYMENT/STORAGE



- + Ensure your transfer chair deployment is smooth
- + No interference when transfer chair is deployed or stored
- + Ensure kickstand extends and retracts correctly

CHECK HEADREST



- + No obvious threading around seams
- + No ripping of headrest material
- + Ensure headrest is free to move along handle

CHECK WHEELS



- + No cracking of wheel or hub
- + No splitting of wheels
- + No malfunction of castor wheel brake
- + Ensure wheels are free to rotate without impediment

CHECK BELTS



- + No deep cracks or rips
- + No large humps or severe bulging

CHECK HAMMOCK



- + No obvious threading around seams
- + No ripping of hammock material
- + Check belt buckle operates correctly



Please contact Paraid®, or an authorised distributor immediately if any of these defects appear on your Transfer chair.

PROMOVE

Products sold separately

ProMove® have been providing innovative transfer slings and accessories for over 15 years, to enable the safe and dignified transfer of individuals that face access barriers.

Committed to creating a more inclusive world, the ProMove Sling range is used as a manual transferring and/or repositioning sling, designed to ensure everyone can be safely transferred from one location to another.

A ProMove Sling is the perfect addition to accompany a Voyager Ambi®, to assist in a safe transfer from one location to another.

CHILD SLING



- + 3 sizes available
- + Age range of 3-18 years
- + Safe working load of up to 20 stone (127kg)
- + Available with or without head support
- + Carry bag included

ADULT SLING



- + Age range of 12+ years
- + Safe working load of up to 45 stone (285kg)
- + Available with or without head support
- + Carry bag included

AIRPORT KIT



- + Includes: adult size sling, extension handles and carry bag
- + Carry bag is fitted with loops and buckles to attach the bag to the Voyager transfer chairs



**View ProMove
sling range**



PARAid®

Your Transfer Specialist

Paraid, a division of Evac+Chair International
Ltd, Unit 4, Central Boulevard, Blythe Valley
Park, Solihull, West Midlands, B90 8AW, UK

+44 (0) 121 700 7455

enquiries@paraid.com

paraid.com