next**RESCUE**

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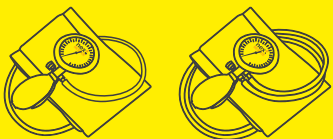
1-Schlauch | 60 mm Zifferblatt
2-Schlauch | 60 mm Zifferblatt

Aneroid Sphygmomanometer

1-tube | 60 mm dial
2-tube | 60 mm dial

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en INSTRUCTION FOR USE

Thank you for purchasing one of our palm-type aneroid sphygmomanometers for professional and home use. This device complies with the international standard ISO 81060-1.

Description:

The blood pressure monitor consists of a manometer, an inflation bulb with an air release valve, and an adult pressure cuff for arm circumferences of 22–32 cm (± 2 cm), which, depending on the model, is connected to the manometer-inflation bulb unit via one or two tubes. The device is latex-free and does not contain batteries.

Intended Use:

The palm sphygmomanometer is a non-automated, mechanical blood pressure monitor used for the indirect, non-invasive measurement and display of arterial blood pressure in humans.

Application:

1. Put the patient into a proper position

It is very important that the person whose blood pressure is being taken is relaxed and sitting comfortably. Keep arm being measured at heart level. Arm extended but not too tense or stiff. Rest arm on a flat surface, such as a desk, with the left palm facing upwards. Keep arm being measured still.

2. How to locate the brachial artery

Place the cuff 2–3 cm above the bend of the elbow on the naked left upper arm. Close the cuff by means of the hook and loop fastener. Cuff should be snug, but not too tight. One or two fingers should fit between cuff and arm. Never place cuff over article of clothing. Palpate the inner arm above the elbow for a pulse. Place the chest piece of the stethoscope over the strongest pulsation.

3. Inflating the cuff

Close the air release valve of the inflation bulb by turning the screw clockwise. Do not over-tighten. Press the inflation bulb evenly until the pressure gauge needle reads 30 mmHg above the expected systolic pressure. If you do

not know the expected systolic pressure, pump until the pressure gauge reads 200 mmHg.

4. Systolic blood pressure reading – upper value

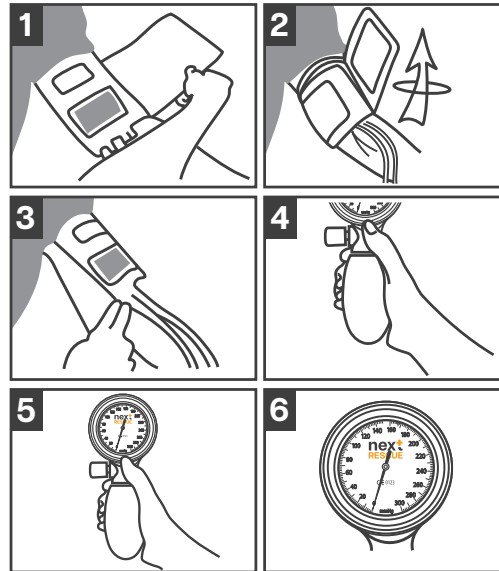
Slowly open the air release valve by turning the screw counterclockwise and hold the stethoscope chest piece over brachial artery. Proper deflation rate is vital for an accurate reading – therefore, you should practice and master a recommended deflation rate of 2–3 mmHg per second or a drop of one to two marks on the pressure gauge per heartbeat. You should not keep the cuff inflated any longer than necessary. As the cuff begins to deflate, you must listen carefully with the stethoscope. Note the reading on the gauge as soon as you hear a faint, rhythmic tapping or thumping sounds. This is the systolic-upper blood pressure reading. Listen carefully and familiarize yourself with pulse sound. Once you are familiar with it, you should check this procedure with your physician.

5. Diastolic blood pressure reading – lower value

Allow the pressure to continue dropping at the same deflation rate. When your diastolic-lower-blood pressure value is reached, the thumping sound stops. Deflate the cuff valve completely. Remove the cuff from arm and the stethoscope from your ears.

6. Record your readings

Repeat the measurement two or more times. Do not forget to record your readings and the time of the measurement immediately after you finish measuring. A convenient time is first thing in the morning, or just before evening meals. Remember that your physician is the only person qualified to analyze your blood pressure.

Possible Side Effects:

Currently, no side effects are known to be associated with the use of this product.

Technical specifications:

- Measuring range: 0–300 mmHg
- Measuring resolution: 2 mmHg
- Accuracy: ± 3 mmHg (0.5 kPa)
- Air leak rate: ≤ 4 mmHg/min
- Pressure reduction rate: 2–3 mmHg/sec
- Storage conditions: -20 °C–50 °C at a relative humidity of 20–85 % (non-condensing)
- Recommended air pressure range: 700–1060 hPa

Taking care of the device:

- Wipe off the manometer and inflation bulb with a damp cloth.
- Cuff: Remove the bladder first, and wipe hook and loop fastener, bladder and tubes with a damp cloth. The cuff can be washed with soap and cold water like all the other cuffs, but you must rinse it with clear water afterwards and let it air dry.

Maintenance:

The end of the product's service life is generally determined by wear and damage from use. Please do not attempt to repair the device yourself. In case of damage or malfunction, contact the manufacturer. Inspect the device regularly for damage to the gauge, tubing, or cuff. Replace the cuff if it shows signs of wear or reduced performance. Only use certified spare parts. Please report any serious incidents to the manufacturer and the competent authority. If applicable (Germany only), please comply with the requirements of the MPBetreibV regarding the performance of metrological control.

Safety instructions:

⚠ Attention!

1. The measurement time should not exceed 2 minutes, with a break of at least 2 minutes between consecutive measurements. Do not place the cuff over wounds to avoid further injury.
2. Do not drop or jar the device.
3. Always deflate cuff completely before storage.
4. Never inflate beyond 300 mmHg.
5. Do not dismantle the manometer under any circumstance.
6. Check the device for damage and cleanliness before each use.
7. Do not use if the device is damaged.
8. Do not expose to extreme temperatures or moisture.
9. Keep away from sharp objects, fire and solvents.
10. Never expose the cuffs to intensive solar radiation.
11. Store complete instrument in storage case provided.
12. Only use original accessories and spare parts.

Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State.

Warranty:

We provide a 3-year quality guarantee for this product from the date of purchase. (Note: The warranty period for the cuff is limited to 1 year.) The receipt serves as proof of the date of purchase. Defects caused by material or manufacturing defects will be remedied free of charge

within the warranty period. The warranty service only extends the warranty period for the replaced components, not for the entire device. Wear and tear, transport damage, and damage caused by improper use (e.g., failure to follow the instructions) or unauthorized intervention are excluded from the warranty. The warranty does not constitute grounds for claims for damages against us. The buyer's statutory rights for defects are not limited by this guarantee.

Disposal:

The device is latex-free and complies with RoHS. It does not contain batteries or electronic components. Please dispose of the device in accordance with local environmental regulations.