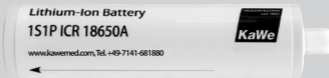
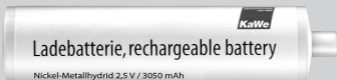
<b>de</b>	Gebrauchsanweisung	Ladebatterien
<b>en</b>	User's manual	Rechargeable batteries
<b>fr</b>	Mode d'emploi	Piles rechargeables
<b>it</b>	Istruzioni per l'uso	Batterie ricaricabili
<b>es</b>	Instrucciones de empleo	Pilas recargables
<b>pt</b>	Manual de operação	Pilhas recarregáveis
<b>ru</b>	Руководство по применению	Аккумуляторы

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## Technical data:

**Note:** In case of using old models of rechargeable batteries manufactured before the publication of this user's manual, special attention should be paid to their technical characteristics (voltage, capacity), which are indicated directly on the battery. They are valid.

REF	12.80110.712***	12.80110.722***	12.80120.712
Type	„2,5V“	„2,5V“	„3,5V“
Voltage [V]	2,5	2,4	3,5
Batterie-Type	nickel metal hydride	nickel metal hydride	nickel metal hydride
Capacity [mAh]	1700	3050	750
Charging time [h]	approx. 8*	approx. 16*	approx. 4*

REF	12.80220.722	12.80120.742
Type	„3,5V“	„3,5V“
Voltage [V]	3,65	3,6
Batterie-Type	lithium-ion	nickel metal hydride
Capacity [mAh]	2850	560
Charging time [h]	approx. 16*	approx. 12**

\* Charging time of fully discharged rechargeable battery using either the KaWe MedCharge 4000.

\*\* Charging time of fully discharged rechargeable battery using an electric wall socket.

\*\*\* It is NOT permitted to use these rechargeable batteries to power 2.5V LED light sources.

**Type:** This designation indicates which lighting fixtures or charging handles the rechargeable batteries are compatible with. For example, 2.5V rechargeable batteries can be used for 2.5V light sources/charging handles (except for 2.5V LED light sources!).

Dear Customer, thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product meets all the requirements of Regulation (EU) 2017/745 (European Union Medical Device Regulation) and, according to this Regulation, is classified as a Class I medical product.

Prior to the use of rechargeable batteries, read this manual thoroughly, paying particular attention to the safety instructions and operation procedure. Familiarize yourself fully with the proper operation of this device before attempting to use it. Save this user's manual for future reference and pass it on to the next user of this device. For questions the connection or operation of this device, please contact customer service.

### **Contraindications for use:**

This device may only be used properly as described here. Noncompliance with the generally applicable regulations and these instructions releases the manufacturer from liability for any resulting damages.

### **Intended use:**

These rechargeable batteries serve as the energy source for the intended KaWe products.

The REF 12.80110.712 and 12.80110.722 rechargeable batteries may not be used for 2.5 V LED lighting equipment.

### **Safety information:**

Improper use or use of damaged batteries may result in the leakage of fumes. Ventilate the area and seek medical attention for health complaints. These fumes can irritate the airways.

Please heed the following instructions:

- Charge the batteries only with the unit intended for this purpose.
- Not allowed: sterilization, autoclave treatment, gas sterilization and Sterrad
- Do not expose to direct sunlight
- Keep away from children

- Store in a cool and dry place
- Never immerse the batteries in liquids.
- After successfully charging the batteries, unplug the charging unit from the wall socket.
- Do not cover the batteries or charging units (with carpet, pillows, bedding etc.)

**Risk of fire! Risk of explosion!**

Never use damaged, defective or deformed batteries. Do not open, damage or drop the batteries. Do not charge the batteries near acids or easily-flammable materials. Protect the batteries from heat and fire. Only use the batteries at an ambient temperature of between 5°C and +40°C. Never lay the batteries on heaters or expose them to direct sunlight for an extended period. After heavy use, first allow them to cool. Do not short-circuit the battery terminals! The batteries are to be wrapped up or packaged (in a plastic bag or box) for disposal, transport or storage. Alternatively, the battery electrodes may also be covered with tape.

**EN****Operating instructions:**

The batteries come only partially charged and must be fully charged before the first use. Use only original KaWe charging handles and charging stations.

**NiMH-rechargeable battery**

If the battery no longer provides enough energy to power your device, turn off the device immediately and charge the battery. (Avoid a deep discharge of the rechargeable battery, as this will destroy it.)

Please note that these batteries will self-discharge. If they are not needed for a longer period of time (used and charged) they will self-discharge, lose capacity and can sustain permanent damage. The battery capacity decreases, and the rechargeable batteries may fail. The rechargeable batteries should therefore be stored with a charge of no less than 50% if they are to be in storage for longer than one month.

Always heed the given charging time, as overcharging will damage the battery. (KaWe MedCharge 4000 charging stations are equipped with an integrated automatic shut-off function.)

NiMH rechargeable batteries lose capacity at low temperatures (less than 0°C).  
When not in use, charge the battery every 3 months.

### Li-ion battery

The Li-ion battery can be recharged at any time without shortening the battery life.  
Interrupting the charging process does not harm the battery.

A significant reduction in the operating time of charged rechargeable batteries indicates that they must be replaced due to aging or defects. Use only original KaWe spare rechargeable batteries. When not in use, charge the battery every 3 months.

### Ambient parameters (during operation/storage/transportation):

Temperature:

from +5 °C to +40 °C

Relative humidity

from 30 to 75 %

### Disposal:

Disposal/recycling instructions are printed on the device itself as well as on the packaging.

Do not dispose of the rechargeable batteries as household waste! They must be brought to special recycling centers.

### Environmentally friendly disposal of batteries:

Batteries must be disposed of properly. Bring defective batteries to a dealer for disposal. Batteries must be removed from devices before they are disposed of. Damaged batteries can be harmful to the environment and your health if they leak poisonous vapors or liquids. Therefore, never mail or otherwise send out a damaged battery. Please consult your local disposal facility/recycling centre. Ensure that the batteries are completely discharged before you dispose of them. We recommend covering the battery contacts with tape in order avoid a short circuit. Do not open the battery.

**Symbol key:**

	Manufacturer		ATTENTION!	
	Date of manufacture		CE conformity label	
	Manufacturer's number, lot number		GOST-R certification of goods imported into Russia	
	Item No.		Temperature range limits	
		Store in dry conditions		Humidity, limit
	Please follow this User's Manual		Separate disposal of electric and electronic devices	
	Separate collection of electrical / electronic equipment and rechargeable batteries		Medical product	

**Warranty:**

No warranties are provided for the rechargeable batteries.

**Contact information:**

Please contact your specialist dealer or directly the manufacturer (see back for address).

**Manufacturer:**

KaWe

**Instructions for service personnel and patients**

In case of problems during the device use, it is necessary to immediately contact the manufacturer and the competent authorities of the relevant member state of the Union, in which the service personnel or the patient are located.

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**KIRCHNER & WILHELM** GmbH + Co. KG