BOSCH
+SOHN

boso

DE

EN

FR

IT

ES

bosso classic

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Contents of the kit

- 1 blood pressure monitor with cuff
- 1 set of instructions for use
- 1 case

Symbols

CE 0124	Device complies with the European Medical Devices Directive
	Year of manufacture
	Manufacturer
mmHg	Unit in which blood pressure is measured
	Proper functionality of the device is tested up to a drop height of 1 metre
	Relative humidity
	Temperature in degrees Celsius
REF	Order number
	Important instructions
	Read instructions for use

Preliminary remarks

This blood pressure monitor complies with the international standard ISO 81060-1.

 Please read this instruction manual carefully before using it and keep in a safe place. All calibration testing – at least every two years – has to be carried out either by the manufacturers or an authorised service technician in accordance with the Medical Devices Directive. Modification to the device and/or the accessories are not permitted. This can lead to measurement errors. The device can be used by any user skilled in auscultatory blood pressure measurement.

Purpose

Non-invasive recording of systolic and diastolic blood pressure in humans. The device can be used for arm circumferences as shown on the cuff.

Important

Protect your blood pressure monitor from dirt and moisture. Make sure not to knock or drop the device and that the cuff is not damaged by sharp objects (needles, scissors etc.).

⚠ Do not inflate to above 300 mmHg.

Individual measurements should not exceed a maximum of 2 minutes. There should be an interval of at least 2 minutes between measurements.

The patient must follow these basic rules when having their blood pressure measured:

⚠ sit comfortably

⚠ not cross their legs

⚠ rest their back and arms on the chair or another surface

⚠ have the centre of the cuff placed on the upper arm so it is level with their heart

⚠ relax as much as possible and not talk while their blood pressure is being measured

⚠ be calm for 5 minutes before the first reading

The following phases are recommended for auscultatory measurement:

⚠ adults: phase V (K5)

⚠ children aged 3 to 12: phase IV (K4)

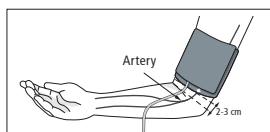
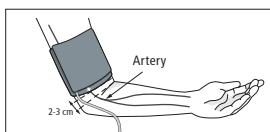
⚠ pregnant women: phase V (K5), except when sounds are audible when the cuff is deflated: in this case phase IV (K4) Korotkoff sounds should be used.

K5 is the point at which tones heard through the stethoscope are no longer audible. K4 is the point at which tones heard through the stethoscope change from a clear sound to a muffled sound.

Fitting the Cuff

⚠ The cuff must not be fitted over wounds as this could lead to further injury.

Blood pressure must always be measured on the arm with the higher pressure values. To do this, first measure blood pressure on both arms and subsequently always on the arm which produced the higher reading.

EN

Place cuff in such a manner that the lower cuff edge finishes approx. 2 – 3 cm above the elbow and the centre of the rubber bag rests on top of the artery.

Taking the measurement

Fully close the valve by turning the air release screw to the right. Inflate the cuff to above the systolic pressure. By turning the air release valve, the air release speed can be adjusted. The World Health Organisation (WHO) recommends a deflation rate of 2 – 3 mmHg per second. To avoid a reading error, read the pressure values perpendicular to the scale.

After the measurement is completed open the release valve to fully deflate the cuff.

Cuffs

Only use the prescribed boso-cuffs with 2-in-1 tube system. The cuff must be selected such that it fits the arm circumference.

Technical Data

Measuring range
0 – 300 mmHg

Accuracy pressure indication
± 3 mmHg

Storage conditions
-30°C to $+70^{\circ}\text{C}$ 15% RH to 85% RH

Operating conditions
-10°C to $+40^{\circ}\text{C}$ 15% RH to 85% RH

Cleaning the device and cuff

- ⚠ Only use a soft, dry cloth to clean your unit.
- ⚠ To clean the cuff, first remove the rubber bag. The outer cuff can then be washed by hand at a maximum temperature of 30 degrees Celsius.

Disinfection

For disinfectant wipes (at least 5 minutes exposure time) of the device and the cuff, we recommend using the disinfectant mikrozid® sensitive liquid (Schülke) to wipe down the device.

We recommend that you spray the cuff with disinfectant.

EN

Warranty Conditions

We provide a 2-year warranty from the date of purchase. The purchase date must be proven by invoice. All material or manufacturing defects will be rectified free of charge within the warranty period. After repairs the warranty period is not extended for the whole unit, but only for the replaced components.

Excluded from the warranty are parts subject to normal wear and tear (e.g. cuff), transport damages and any damage caused by improper handling (e.g. non-compliance with the instruction manual for use). Damages due to disassembly by unauthorized persons are also excluded from warranty.

No claims for damages against us are substantiated by the warranty. In the case of justified warranty claims the device must be sent along with the original invoice to:

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Disposal

Please ensure that the device is disposed of according to all regional and national environmental regulations.

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BOSCH + SOHN GmbH u. Co. KG