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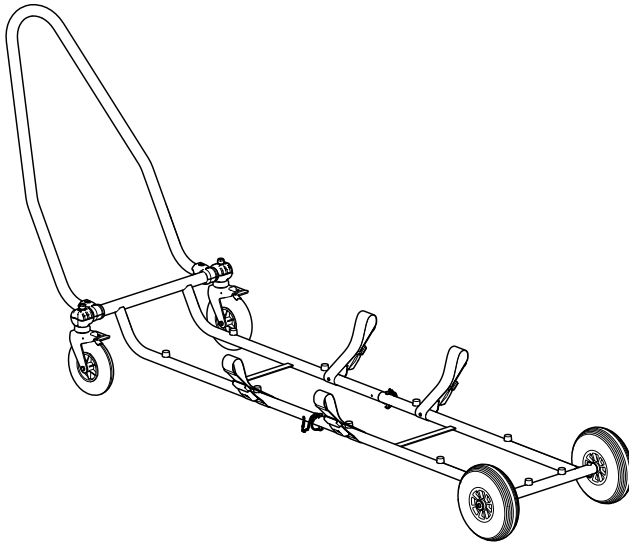
EN

1.	MODELS	3
2.	INTENDED USE	3
2.1	INTENDED USE AND CLINICAL BENEFITS	3
2.2	TARGET PATIENT	3
2.3	PATIENT SELECTION CRITERIA	3
2.4	CONTRAINDICATIONS AND UNWANTED SIDE EFFECTS	3
2.5	USERS AND INSTALLERS	3
3.	REFERENCE STANDARDS	3
4.	INTRODUCTION	3
4.1	USING THE MANUAL	3
4.2	DEVICE LABELLING AND TRACEABILITY	4
4.3	SYMBOLS	4
4.4	WARRANTY AND SERVICE	4
5.	WARNINGS	4
6.	SPECIFIC WARNINGS	5
7.	RESIDUAL RISK	6
8.	TECHNICAL DATA AND COMPONENTS	6
9.	COMMISSIONING	6
10.	OPERATING CHARACTERISTICS	7
11.	PROPER USE	7
12.	CLEANING AND MAINTENANCE	8
12.1	CLEANING	8
12.2	ROUTINE MAINTENANCE	8
12.3	PERIODIC OVERHAUL	9
12.4	SPECIAL MAINTENANCE	9
12.5	LIFE SPAN	9
13.	TROUBLESHOOTING TABLE	9
14.	ACCESSORIES	9
15.	SPARE PARTS	9
16.	DISPOSAL	9

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1. MODELS

The following basic models may be subject to implementation or change without notice.

- ST04530 SPEED BASKET TRANSPORT CARRIER

2. INTENDED USE

2.1 INTENDED USE AND CLINICAL BENEFITS

The basket stretcher carrier is a device designed for the support and ground handling of basket stretchers that are especially suitable for use in the field as they allow the stretcher to be moved without lifting it, reducing strain on the operators and facilitating patient handling.

2.2 TARGET PATIENT

The target patients are those for whom use of a basket stretcher is foreseen.

2.3 PATIENT SELECTION CRITERIA

Patient selection is related to the use of the basket stretcher.

2.4 CONTRAINDICATIONS AND UNWANTED SIDE EFFECTS

No particular contraindications or side effects are known with relation to use of the device, as long as it is used in accordance with the user manual.

2.5 USERS AND INSTALLERS

The intended users are those foreseen for use of the basket stretcher and persons trained in first aid procedures and in use of medical equipment in an emergency medical service (EMS) environment.

Basket stretcher carriers are devices intended for professional use only. Do not allow untrained persons to help while using the product, as they may cause injury to themselves or others. Despite all efforts, laboratory tests, trials, and instructions for use, standards do not always reproduce practice, so the results obtained under actual conditions of product use in the natural environment may sometimes differ significantly.

The best instructions are the continuous practice of use under the supervision of competent and trained personnel.

Operators using the device should be physically able to use the device and have good muscle coordination, as well as strong back, arms, and legs, should it be necessary to lift and/or support the device and the patient. Operators' ability must be assessed before the definition of roles in use of the stretcher.

Operators must be able to provide the necessary patient care.

■ 2.5.5 USER TRAINING

- Regardless of your level of experience with similar devices in the past, you should carefully read and understand the contents of this manual before installing, operating, or servicing this product. In case of any questions, please contact Spencer Italia S.r.l. for the necessary clarifications.
- The product must be used only by personnel trained in the use of this product and not on other similar products.
- The suitability of the users for use of this product can be attested by the training registration, in which trained persons, trainers, date and place are specified. **This documentation must be kept for at least 10 years after the end of the product's life and must be made available to the competent authorities and/or the Manufacturer when requested. In the absence of such documentation, the relevant bodies will apply any foreseen sanctions.**
- Do not allow untrained persons to help while using the product, as they may cause injury to themselves or others.
- The product must be put into use only by personnel trained in the use of this product and not on other similar products.

Note: Spencer Italia S.r.l. is always available for training courses.

■ 2.5.6 INSTALLER TRAINING

Installation is not required.

3. REFERENCE STANDARDS

As Distributor or End-User of the products manufactured and/or marketed by Spencer Italia S.r.l., users are strictly required to be familiar with the legal provisions in force in the country of destination of the goods, applicable to the devices to be supplied (including regulations relating to technical specifications and/or safety requirements) and, therefore, to understand the requirements necessary to ensure compliance of the products themselves with all legal requirements of the territory.

REFERENCE	DOCUMENT TITLE
EU Regulation 2017/745	EU Regulation on Medical Devices

4. INTRODUCTION

4.1 USING THE MANUAL

The purpose of this manual is to provide healthcare professionals with the information necessary for safe and appropriate use and maintenance of the device.

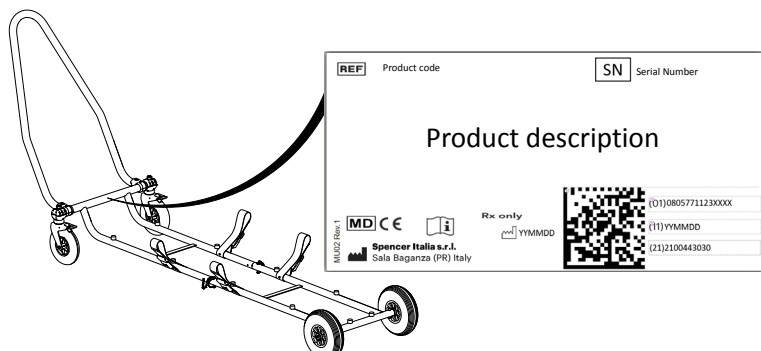
Note: The Manual is an integral part of the device and therefore it must be kept for the entire life of the device and must accompany it in any changes of use or ownership. If any instructions for use for products other than the one received are present, please contact the Manufacturer immediately before use.

Spencer product User Manuals can be downloaded from the website www.spencer.it/en/emergency-medical-solutions/user-manuals or by contacting the Manufacturer. Exceptions are those items whose essentiality and reasonable and predictable use are such that it is not necessary to draw up instructions, in addition to the following warnings and indications on the label.

Regardless of your level of experience with similar devices in the past, it is advisable to carefully read and understand the contents of this manual before installing, operating, or servicing this product.

4.2 DEVICE LABELLING AND TRACEABILITY

Each device is provided with a label, placed on the device itself and/or on the packaging, which contains the Manufacturer's identification data, product, CE marking or lot number (LOT). This must never be removed or covered.

In the event of damage or removal, request a duplicate from the Manufacturer, or else the warranty will be void as the device can no longer be traced.

EU Regulation 2017/754 requires manufacturers and distributors of medical devices to keep track of their location. If the device is in a location other than the address to which it was shipped or sold, or if it was donated, lost, stolen, exported or destroyed, permanently removed from use, or if the device was not delivered directly by Spencer Italia S.r.l., inform Customer Service (see § 4.4).

4.3 SYMBOLS

Symbol	Meaning	Symbol	Meaning
CE	Device in compliance with EU Regulation 2017/745	⚠	Danger – Indicates a hazardous situation that may result in a situation directly related to serious injury or death.
MD	Medical device	📖	See the user manual
🏭	Manufacturer	LOT	Serial Number
📅	Date of manufacture	REF	Product code
UDI	Unique Device Identifier		

Production identification
Alphanumeric code that identifies the production units of the device, composed of:

(01)0805771123XXXX	Company prefix (XXXX=GTIN)
(11) YYMMDD	Date of manufacture
(NN) 1234567890	(NN)1234567890 NN=10 =>LOT/ NN=21=>SN

4.4 WARRANTY AND SERVICE

Spencer Italia S.r.l. guarantees that products are free from defects for a period of **one year from the date of purchase**.
For information regarding correct interpretation of the instructions for use, maintenance, installation or return, please contact Spencer Customer Service tel. +39 0521 541154, fax +39 0521 541222, e-mail service@spencer.it
To facilitate service, always indicate the lot number (LOT) or serial number (SN) on the label attached to the package or device itself.

Warranty and service conditions are available at www.spencer.it/supporto/termini-e-condizioni

Note: Record and keep with these instructions: lot (LOT) or serial number (SN), if present, place and date of purchase, date of first use, date of checks, user name and comments.

5. WARNINGS

⚠ Warnings, notes, and other important safety information are provided in this section and are clearly visible throughout the manual.
At least every 6 months, it is important to check for updated instructions and any changes involving your product. This information is freely available on the website www.spencer.it on the specific product page.

Product features

- ⚠ Use of the product for any purpose other than that described in the instructions for use is prohibited.
- Before each use, always check the conditions of the product, as specified in the User Manual. In the event of faults/damage that could compromise its functionality/safety, immediately remove it from service and contact the Manufacturer.
- If the product is found to be malfunctioning, immediately use a similar device to ensure continuity of ongoing operations.
- ⚠ The product must not be tampered with or modified without the manufacturer's authorisation (modification, tweaking, additions, repair, use of non-approved accessories), as they may constitute imminent danger of injury to persons and material damage. Should these operations be performed, we decline any responsibility for incorrect operation or any damage caused by the product itself; moreover, the CE marking (when required by law) and the product warranty shall be null and void.
- When using the devices, position and adjust them in such a way that they do not hinder operator works or the use of any other equipment.
- When using the devices, position and adjust them in such a way that they do not hinder operator works or the use of any other equipment.
- Be sure to take every precaution to avoid hazards from contact with blood or body secretions, if applicable.
- Always comply with the maximum capacity indicated in the instructions for use.

- Avoid contact with sharp or abrasive objects.
- Operating temperature: from -5°C to + 50°C.

Storage

- The product must not be exposed or come into contact with thermal sources of combustion or flammable agents, but must instead be stored in a dry, cool place, away from light and sun.
- Do not store the product under other more or less heavy materials that may damage the structure of the product.
- Store and transport the product with its original packaging, otherwise the warranty shall be invalidated.
- Storage temperature: -10°C to +60°C.

Regulatory requirements:

As Distributor or End-User of the products manufactured and/or marketed by Spencer Italia S.r.l., users are strictly required to be familiar with the legal provisions in force in the country of destination of the goods, applicable to the devices to be supplied (including regulations relating to technical specifications and/or safety requirements) and, therefore, to understand the requirements necessary to ensure compliance of the products themselves with all legal requirements of the territory.

- Promptly and in detail notify Spencer Italia S.r.l. (already in the quotation request phase) about possible fulfilments by the Manufacturer necessary for the compliance of products with specific legal requirements of the territory (including those deriving from regulations and/or regulatory provisions of another nature).
- Act with due care and diligence to help ensure compliance with the general safety requirements of the devices placed on the market, providing end-users with all the information necessary to carry out periodic revisions on the supplied devices, exactly as indicated in the User Manual.
- Participate in safety checks on products placed on the market**, transmitting information regarding product risks to the Manufacturer as well as to the Competent Authorities for their respective actions.
- Without prejudice to the above, the Distributor or End-User shall assume wider liability related to non-compliance with non-fulfilment of the above-mentioned obligations, with consequent obligation to indemnify and/or hold Spencer Italia S.r.l. harmless from any possible injurious effect.
- With reference to EU Regulation 2017/745, please note that public or private operators who, when exercising their activity, detect an incident involving a medical product are required to notify the Ministry of Health, within the terms and in the manner established by one or more ministerial decrees, and notify the Manufacturer. Public or private health care professionals are required to notify the Manufacturer of any other incident that may allow the adoption of measures to ensure the protection and health of patients and users.

General warnings for medical devices

The user must carefully read the following in addition to the general warnings.

- Application of the device should not last longer than the time associated with use of the basket stretcher.
- ⚠ Qualified personnel and at least two operators must be present during use of the device.
- Do not use if the device or parts of it are damaged or excessively worn.
- Follow the internal procedures and protocols approved by your organisation.
- ⚠ Do not alter or modify the device arbitrarily, as doing so could result in unpredictable operation and damage to the patient or rescuers and shall void the manufacturer's warranty and release the manufacturer from all liability.
- Disinfection operations must be carried out in accordance with the validated cycle parameters, as stated in the specific technical standards.
- Do not use drying machines to dry the device.

6. SPECIFIC WARNINGS

To use the device, you must also have read, understood and carefully follow all the instructions in the instructions for use.

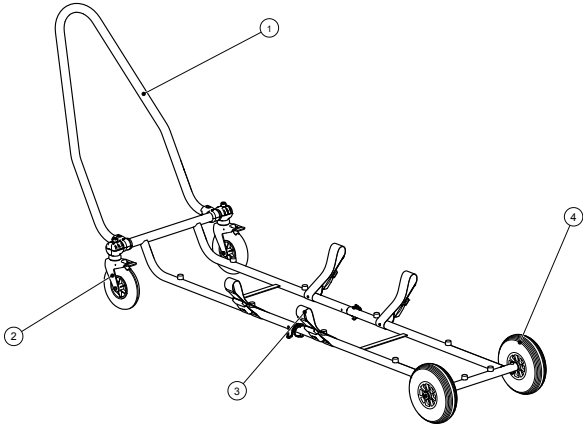
- ⚠ Always comply with the maximum capacity, if any, indicated in the instructions for use.
- Make sure that operators are in good physical condition before handling, as listed in the instructions for use.
- The maximum weight, which weighs on each operator, must comply with local health and safety requirements.
- Establish a maintenance program and periodic checks, identifying a designated reference person. The person entrusted with routine maintenance of the device must ensure the basic requirements envisaged by the manufacturer within these operating instructions.
- All maintenance activities must be recorded and documented with the relevant technical operation reports. This documentation must be kept for at least 10 years after the end of the device's life and must be made available to the competent authorities and/or the manufacturer when requested.
- Use only original or Spencer Italia S.r.l. approved components/replacement parts and/or accessories to carry out any operation without causing alterations or modifications to the device. Otherwise, we decline all responsibility regarding incorrect operation or any damage caused by the device to the patient or the operator, invalidating the warranty and invalidating compliance with EU Regulation 2017/745.
- Never leave the patient on the device unsupervised, as they could get injured.
- Avoid contact with sharp objects.
- Follow approved Emergency Medical Service procedures for patient immobilization and transportation.
- Follow approved Emergency Medical Service procedures for patient positioning and transportation.
- ⚠ Before lifting, make sure that operators have a secure grip on the device.
- The device is an accessory for supporting basket stretchers and cannot be used as a stationing device.
- Practice with an empty stretcher to make sure you are familiar with the manoeuvres.
- ⚠ At least two operators in suitable physical conditions are required for use of the device. They must be endowed with strength, balance, coordination, common sense and must be trained on the correct operation of the Spencer stretcher device.
- For patient loading techniques for particularly heavy patients, for operations on steep terrain or in special and unusual circumstances, the presence of more than two operators is recommended in addition to the two minimum operators.
- ⚠ Before each use, always check the conditions of the device and its components and accessories, as specified in the user manual. In case of faults or damage that may compromise the functionality and safety of the device, and therefore the patient and the operator, remove the device from service or replace the components that are not intact.
- Make sure that the stretcher is correctly locked onto the device.
- Do not move the stretcher if the weight is not properly distributed.
- ⚠ Pay the utmost attention to any obstacles (water, ice, debris, etc.) present on the route, as they may cause the operator to lose balance and compromise proper functioning of the device. If you cannot clear the route, choose an alternative route.
- Do not leave the patient without the assistance of at least two operators when the medical device is in use.

7. RESIDUAL RISK

No residual risks, or rather risks that could arise despite compliance with all warnings in this user manual, have been identified.

8. TECHNICAL DATA AND COMPONENTS

Note: Spencer Italia S.r.l. reserves the right to make changes to specifications without notice.

N°	Description	Material	N°	Description	Material
1	Manoeuvring handle	Aluminium	3	Basket fixing strap with metal buckle	Steel/PU
2	Front wheel with brake	Steel/PU	4	Rear wheel	Polyurethane

Specifications	Value
Length (Not including Handlebar) [mm]	2125
Width [mm]	710
Dimensions when folded [mm]	1225 x 710 x 415
Weight [kg]	15
Front wheel diameter [mm]	200
Rear wheel diameter [mm]	260
Load capacity [kg]	300
Materials	Aluminium, Steel, Nylon

Note: The data shown does not take into account any accessories in use and/or installed on the device.

9. COMMISSIONING

For first use, check that:

- Packaging is intact and has protected the device during transportation
- Check that all parts included in the packing list are present
- General functionality of the device
- The fastenings must keep the basket stretcher and the support carrier attached.
- There are no cuts, holes, tears or abrasions on the entire device.

Do not modify the device or any of its parts for any reason as this could cause damage to the patient and/or rescuers.

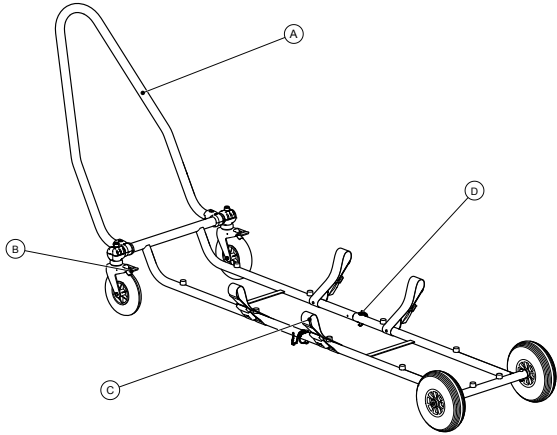
⚠ Failure to take the above measures will preclude safe use of the device, resulting in risk of damage to the patient, operators and the device itself.

For subsequent use, perform the operations specified in paragraph 12.

If the above conditions are met, the device may be considered ready for use; otherwise, you must immediately remove the device from service and contact the Manufacturer.

Do not alter or modify the device arbitrarily, as doing so could result in unpredictable operation and damage to the patient or rescuers and will void the warranty and release the Manufacturer from all liability.

10. OPERATING CHARACTERISTICS

Element	Description	Function
A	Manoeuvring handle	Allows easy manoeuvring of the device
B	Brake	Located on the front wheel brackets; brakes the wheels when activated
C	Fixing strap	Secures the basket stretcher to the Speed Basket carrier device
D	Connecting pin	Allows the frame to be divided into two parts for easy storage of the device

See paragraph 11 - Proper use for operating characteristics.

11. PROPER USE

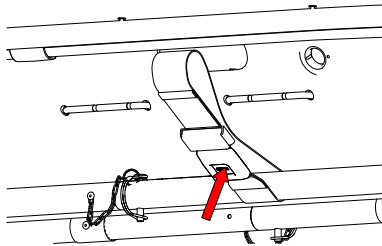
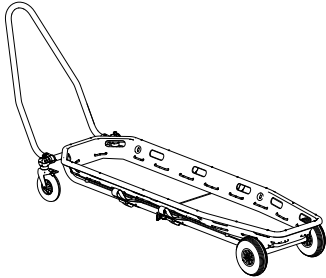
Primary medical evaluations must be carried out before the patient is moved, lifted or transported. Before loading the basket stretcher on the carrier, bring it as close as possible to the device.

1. LOCKING THE BASKET STRETCHER

The procedure for securing the Speed Basket device is described below.

Position the basket stretcher on the Speed Basket carrier device, making sure that the weight is properly distributed and balanced to minimise the strain on operators during transport.


Press the button to release the belt buckle and secure the basket stretcher on the Speed Basket carrier device.


Two operators are required to handle the device.

2. STORING THE SPEED BASKET DEVICE

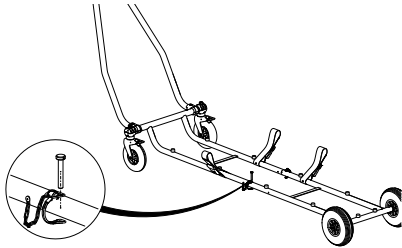
Remove the connecting pin so that the front part of the frame can be separated from the rear part.

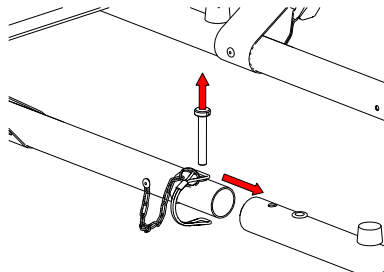
Separate the rear frame from the rest of the frame of the carrier device.

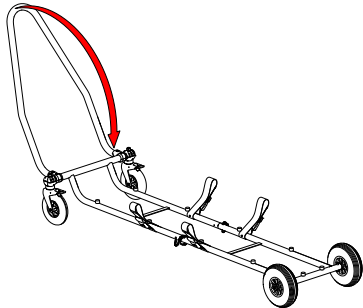Close the manoeuvring handle in order to reduce the size of the device.

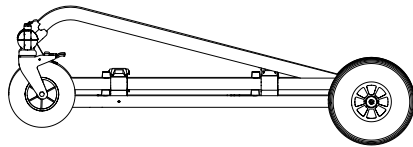Once disconnected, place the rear frame below the front frame, so that the device is more compact and ready for storage.

12. CLEANING AND MAINTENANCE

Spencer Italia S.r.l. declines all responsibility for any direct or indirect damage which is the consequence of improper use of the product and spare parts and/or in any case of any repair carried out by a person other than the Manufacturer, who uses internal and external technicians authorised; to do so; moreover, doing so will invalidate the warranty.

- The operator must wear suitable personal protective equipment, such as gloves, goggles, etc. during all checking, maintenance and cleaning operations.
- Establish a maintenance schedule, periodic inspections and extend the average life span, if foreseen by the Manufacturer in the User Manual, identifying a reference person who meets the basic requirements set forth in the User Manual.
- The frequency of inspections is determined by factors such as legal requirements, type of use, frequency of use, and environmental conditions during use and storage.**
- Repairs of products manufactured by Spencer Italia S.r.l. must be carried out by the Manufacturer, who shall make use of specialised internal or external technicians who, using original spare parts, shall provide quality repair service in strict compliance with the technical specifications indicated by the Manufacturer. Spencer Italia S.r.l. declines any responsibility for any direct or indirect damage resulting from improper use of spare parts and/or any repair work carried out by unauthorised parties.
- Reconditioning, a process performed on the device to restore the technical and functional safety of the device used, for example re-registration, must be performed by the Manufacturer.
- Where foreseen, use only original or Spencer Italia S.r.l. approved components/spare parts and/or accessories in order to carry out all operations without causing alterations or modifications to the product.
- All maintenance and overhaul activities must be recorded and documented with the relevant technical operation reports. This documentation must be kept for at least 10 years after the end of the product's life and must be made available to the competent authorities and/or the Manufacturer when requested.
- Cleaning, provided for reusable products, must be carried out in accordance with the Manufacturer's instructions in the User Manual in order to avoid the risk of cross-infection due to the presence of body fluids and/or residues.
- If required, the product and all its components must be washed and left to dry completely before storage.
- If the product requires lubrication, this must be done after cleaning and complete drying.

12.1 CLEANING

Failure to carry out the correct cleaning operations could increase the risk of cross-infection due to presence of body fluids and/or residues.
The operator must wear suitable personal protective equipment, such as gloves, goggles, etc. during all checking and cleaning operations.

Any metal parts exposed to external agents undergo surface treatments and/or coating in order to obtain better resistance. Clean the exposed parts with water and delicate soap. **Never use solvents or stain removers.**

Rinse thoroughly with lukewarm water, making sure you have removed all traces of soap, which may deteriorate or compromise conditions and durability. **Avoid using high pressure water**, as it penetrates the joints and removes lubricant, creating the risk of corrosion on components. Let dry completely before storing. Drying after washing or after use in a wet environment must be natural and not forced. Do not use flames or other direct heat sources.

If **disinfecting**, use products that do not have a solvent or corrosive action on materials constituting the device, in addition to being classified as medical-surgical devices. Be sure to take all precautions to ensure that there is no risk of cross-infection or contamination of patients and operators.

In the case of disposable products, no cleaning is required except that the product be properly stored and packaged according to the manufacturer's specifications.

12.2 ROUTINE MAINTENANCE

The person entrusted with device maintenance must ensure the basic requirements envisaged within this user manual.
All routine and special maintenance activities and all general overhauls must be recorded and documented with the relevant technical intervention reports. This documentation must be kept for at least 10 years after the end of the device's life and must be made available to the competent authorities and/or the Manufacturer when requested.

The operator must wear suitable personal protective equipment, such as gloves, goggles, etc. during all checking, maintenance and cleaning operations.

The device does not require a routine maintenance program, but checks must be made to verify:

- General functionality of the device
- Cleanliness of the device (remember that the failure of cleaning may cause the risk of cross infections)
- Fulfillment of the requirements of the user manual in section 5 Warnings and 6 Specific Warnings.
- Fulfillment of the requirements of the manual in section 11 Proper use.

These checks must be carried out before and after each use.

Use only original or Spencer Italia S.r.l. approved components/replacement parts and/or accessories to carry out any operation without causing alterations or modifications to the device. Otherwise, we decline all responsibility regarding incorrect operation or any damage caused by the device to the patient or the operator, invalidating the warranty and invalidating compliance with EU Regulation 2017/745.

12.3 PERIODIC OVERHAUL

No periodic overhaul is foreseen for the device.

12.4 SPECIAL MAINTENANCE

Special maintenance can only be carried out by the Manufacturer, who uses internal and external technicians specialised and authorised by the Manufacturer itself.

Only maintenance activities carried out by specialised technicians authorised by the Manufacturer are considered valid by Spencer Italia S.r.l.

The end-user can replace only the spare parts indicated in § 15.

12.5 LIFE SPAN

The device, if used as described in the following instructions, has a life span of 5 years from the date of purchase.

Spencer Italia S.r.l. will accept no responsibility for incorrect operation or damage caused by the use of devices that have exceeded the maximum allowable life span.

13. TROUBLESHOOTING TABLE

PROBLEM	CAUSE	REMEDY
The locking mechanisms are not working properly	The adjustment/fixing systems are damaged	Immediately remove the device from service and contact the manufacturer
The wheel does not slide	Wheel damaged	Replace the wheel
The basket stretcher appears unstable when being placed on the device	Non-horizontal ground, unbalanced weight or improperly positioned stretcher	Check correct positioning of the stretcher

If a problem or fault is detected that does not correspond to the above, please contact Spencer Italia srl customer care service.

14. ACCESSORIES

There are no accessories available.

15. SPARE PARTS

CODE	DESCRIPTION
ZC10113	SOLID BLACK WHEEL Ø260 FOR JACK W/BLACK DISC
RIBA017	BLACK WHEEL 2 Ø200 WITH BEARING
RIBA023	FE CHROME BRACKET W/WHEEL BRAKE Ø200 W/HEX

16. DISPOSAL

When devices and their accessories are no longer suitable for use, they can be disposed of as normal municipal solid waste if they have not been contaminated by special agents. Otherwise, follow the regulations in force regarding disposal.

Notice

The information contained in this document is subject to change without notice and is to be intended as a commitment by Spencer Italia S.r.l. subject to change.
The images are included as examples and may vary slightly from the actual device.

© Copyright Spencer Italia S.r.l.

All rights reserved. No part of this document may be photocopied, reproduced or translated into another language without the prior written consent of Spencer Italia S.r.l.

Prima emissione: 10/02/2025
Rev: 0
Codice CCI5367

First issue: 10/02/2025
Rev: 0
Code CCI5367