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ultraSAVER SPEZIAL / ultraSAVER SEZIAL-XXL

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ultraSAVER SPEZIAL

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kg 1,4 kg **max.** 400 kg **E** 200 x 70 cm

ultraSAVER SPEZIAL-XXL

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kg 4,8 kg **max.** 700 kg **E** 220 x 130 cm

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 4,8 kg  700 kg  220 x 130 cm

Intended use of the product

The ultraSAVER SPEZIAL /-XXL is a medical device for evacuating bedridden or injured people.

User groups

According to § 2, paragraph 2 MPBetriebV, this product may only be used by competent, trained specialists. The prerequisite is theoretical and practical knowledge of how to use the rescue sheet.

Important information

Before using the rescue sheet, please read the instruction for use and safety instructions carefully. Keep the instruction for use in a safe place to avoid operating errors.

Declaration of conformity

This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

General safety instructions

Check the integrity of the rescue sheet **before and after** each use. If damage or only suspicion of damage occurs, take the product out of operation immediately.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

Use

The ultraSAVER SPEZIAL/-XXL should only be used by trained personnel. When lifting patients with more than 2 users, make sure that an even tension is built up on the rescue sheet.

Storage

Store the rescue sheet in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

Maintenance and Care

The rescue sheet can be used several times. Disinfection should be carried out according to the RKI recommendation "surface disinfection after patient contact" or with commercially available disinfectants.

To ensure the safety of the patient and the user, we recommend that the rescue sheet be subjected to **an annual** visual inspection by an expert.

For users in Germany, we recommend observing the following regulations and guidelines: DGV Principle 305-002 / DGV Rule 112-198/199 / MPBetriebV.

Service life

The service life of the ultraSAVER SPEZIAL/-XXL depends on the individual conditions of use. Depending of the frequency and intensity of use certain wear and tear is normal. Appropriate use is assumed here.

The material is subject to aging even when handled carefully. This is accelerated by UV rays, heat, chemicals and dirt. The product should be phased out depending on the condition or at the latest after 10 years.