HUM

Systems for Life®

AEROresc® – EASY Splint

REF: HIMO2-SP-XL | HIMO2-SP-M | HIMO2-SP-XS | HIMO2-SP-XL-SET | HIMO2-SP-M-SET | HIMO2-SP-SET1 | HIMO2-SPT-XL | HIMO2-SPT-M | HIMO2-SPT-XS | HIMO2-SPT-XL-SET | HIMO2-SPT-M-SET | HIMO2-SPT-SET1

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en INSTRUCTION FOR USE

Intended Use: It is physical supports for injured patients to secure or support patients in first aids or physical therapy. The Medical AEROresc® – EASY Splint is worn on the patients' upper or lower

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extremities to support, correct and prevent deformities, or align body structures for functional improvement.

Application: The medical AEROresc® – EASY Splint is the emergency splint of choice for: First Responders, EMS, Military Medics, Athletic Trainers, Wilderness Rescue, Ski Patrols, Outdoor Adventurers, Safety Engineers, Veterinarians, and more.

Directions for use:
1. Make the medical AEROresc® – EASY Splint to suitable shape to fracture part.
2. Fixation together fracture part and medical AEROresc® – EASY Splint with bandage or straps.

Features:

- Lightweight and compact
- Quickly immobilize an injured limb
- Waterproof and Radiolucent
- Latex Free

Cleaning and Disinfection:
1. Clean the splint with warm, soapy water and a cloth or soft brush, or wipe with a mild solvent after each use, and dry it in air.
2. Wipe all surfaces with a disinfectant (for example, Accel® PREvention™ Wipes and RTU). Please follow disinfectant manufacturer’s instructions.

Storage: Avoid high temperature & moisture.

Shelf life: Five years

Safety Instructions

⚠ Attention!

1. The instruction for use does not replace detailed instructions / training of the user. Please use under the guidance of a healthcare professional.
2. The AEROresc® – EASY Splint may only be used for the intended use described in these instructions for use.
3. Do not use for fractures of the spine and pelvis.
4. Do not use on open wounds. Open wounds must be covered with sterile dressings before using the AEROresc® – EASY Splint.
5. Cutting the AEROresc® – EASY Splint might result in sharp edges that need to be covered or folded.
6. For prolonged treatment duration (> 1 hour) a patient-side cushion pad should be used.
7. When fixing the AEROresc® – EASY Splint, make sure that the blood flow is not interrupted and that the tips of fingers or toes are exposed to check the peripheral circulation.

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