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SCHAUFELTRAGE REF **N5 1000**

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INSTRUCTIONS FOR USE

SCOOP STRETCHER **REF N5 1000**

Please read the Instructions for use carefully before using the scoop stretcher

APPLICABLE SCOPE

The scoop stretcher was developed for professional use by rescue services, disaster control and the military by respectively trained personnel.

FEATURES

1. Made of aluminum alloy, the scoop stretcher consists of a rigid structure that can be opened at both ends and thus separated into 2 parts. This minimizes the need to move the patient on site. The stretcher is used to quickly pick up and put down the patient to reduce secondary injuries.
2. Concave surface for safe positioning of the patient.
3. The length of the stretcher can be adjusted to three different positions based on the size of the patient.
4. It is characterized by being lightweight, safe to use and easy to sterilize.
5. The two plates on the head side can be used to attach a head fixation there.

TECHNICAL SPECIFICATION (TOLERANCE $\pm 5\%$)

Model	
Unfolded Size (L x W x H)	200 x 42 x 7 cm
Folded Size (L x W x H)	165 x 42 x 7 cm
Self-weight	8.7 kg
max. load	159 kg

USE METHOD

1. Length adjustment

Turn the two side locking knobs (Fig. 1) and stretch or shorten the length (Fig. 2). of the scoop stretcher. The length can be adjusted in three different positions, each position corresponds to one hole, select the appropriate position, release the knob when horizontal, hold the locking knob until the locking is complete.

Hole 0 is the home position, hole 1 is position 1, hole 2 is position 2, hole 3 is position 3

(Note: positions 1 and 2 are for normal use). The length of the initial position 0 = 167cm, position 1= 182cm, position 2 = 196cm, position H3 = 211cm. (Fig. 3)

⚠ WARNING: Adjust the length of the stretcher so that you do not go beyond the 3rd position.

Fig. 1

Fig. 2

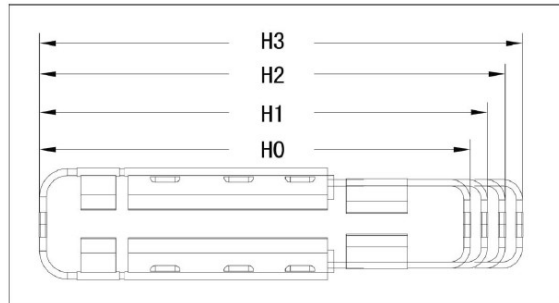
Fig. 3

2. Press the clutch button at both ends of the stretcher so that the stretcher can be separated into two parts. Slide both parts under the patient from the side and close the locking heads again. (Fig. 4 and 5).

⚠ WARNING: Make sure that the locking button is properly engaged before transporting the patient.

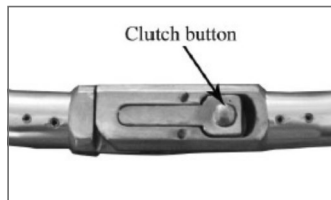
Fig. 4

Fig. 5

3. The scoop stretcher can be used for immobilization of the head with a head fixation. (Fig. 6).
4. After use, it can be stored folded. To do this, bring the scoop stretcher to its maximum length before hand (Fig. 7).

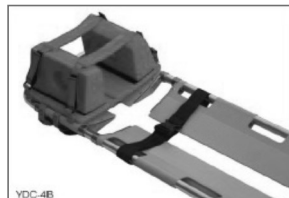
Fig. 6

Fig. 7

NOTES

1. When transporting, make sure the locking knobs on both ends are locked, use the safety belts.
2. After adjusting the length, make sure the position is locked. Please do not use a stretcher that is not locked, so as not to injure the patient. When you adjust the length of the stretcher, do not go beyond the 3rd position.

MAINTENANCE

1. Routine cleaning (incl. disinfection).
2. Check for loose or altered parts.

STORAGE

1. Store the scoop stretcher in a moisture-proof and corrosion-free environment.
2. Store, transport as indicated on the packaging.

TRANSPORT

1. By airplane, ship or road.
2. The stretcher can be moved by hand.

PACKING LIST

Wrapped in nylon bag and packed in durable cardboard box.

Accessories: safety belts (3 pieces)

Erstellt am/Created on: 2023-12-19

 JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.,No.427 Yangjin Road, Jinfeng,
Zhangjiagang, Jiangsu Province, CHINA

 SUNGO Europe B.V., Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, The Netherlands

 servoprax GmbH, Am Marienbusch 9, 46485 Wesel, Germany,
[www. Servoprax.de](http://www.Servoprax.de), info@servoprax.de

1 - N5 1000 - 075 - 2 - 0002 - 2312