Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

DH-40
DH-40 L


DE

EN

ES

FR

Enjoy mobility.

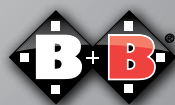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[illegible]

Index

1.	Preliminary remark	28
1.1.	Signs and symbols	28
2.	Important safety instructions	29
2.1.	Indication	30
2.2.	Contraindication	30
2.3.	Declaration of Conformity	31
2.4.	Responsibility	31
3.	Intended use	31
4.	Product and delivery overview	32
4.1.	Inspection of the delivery	32
4.2.	Scope of delivery	32
4.3.	Overview	33
4.3.1.	Shower aid with backrest	33
4.3.2.	Shower aid without backrest	34
4.4.	Type plate and serial number	35
5.	Assembly and customisation	36
5.1.	Assembly	36
5.2.	Mounting the legs	36
5.3.	Mounting the backrest	37
5.4.	Adjusting the seat height	37
6.	Check before use	38
6.1.	Using the shower aid	38
7.	Technical data	40
8.	Care instructions for the user	41
8.1.	Cleaning	41
8.2.	Disinfection	42
9.	Maintenance	43
9.1.	Maintenance instructions	43
9.2.	Maintenance instructions for the specialised trade	43
10.	Repair	44
11.	Storage and dispatch	44
12.	Waste disposal	45
13.	Passing on reuse	45
14.	Useful life	45
15.	Warranty	46

1. Preliminary remark

Dear customers,

You have decided in favour of a high-quality shower aid from Bischoff & Bischoff. Thank you for placing your trust in us. This operating instruction contains all the necessary information for installation, operation and care. Please read and follow this operating instruction, including all safety instructions, carefully before using the product for the first time.

The DH-40 shower aid is a height-adjustable shower aid with handles. The DH-40 L shower aid also has a backrest, but is otherwise identical in construction to the DH-40.

This instruction is an integral and necessary part of the product. Keep the operating instruction to hand and pass them on when passing on the product. For people with visual impairments, this document is available as a PDF file on our website www.bischoff-bischoff.com.

1.1. Signs and symbols

Important! Indicates particularly useful information in the respective context.

Attention! Indicates particularly safety-relevant information. Observe the operating instructions!

2. Important safety instructions

- Max. user weight of the DH-40 and DH-40 L is 110 kg.
- Only use the shower aid at home, if necessary with the support of an assistant.
- To avoid falls and dangerous situations, you should be instructed in the use of the shower aid by a specialist dealer or carer.
- Before using the shower aid, please check that all attached parts are correctly fastened.
- All four legs must be set to the same length.
- Make sure that the shower aid stands securely on the rubber feet.
- Be aware of the risk of pinching when setting up.
- The seat is only suitable for sitting on.
- Avoid leaning out far to the side. Danger of tipping!
- The shower aid must be secured against unauthorised use!
- No additions or modifications may be made to the product. The commode chair may only be used with accessories approved by Bischoff & Bischoff.
- Exposure to sunlight can cause the surfaces of the shower aid to become very hot and lead to burns. Therefore, avoid direct sunlight.

Caution! Parts heated by sunlight can cause skin injuries.

- At low ambient temperatures, metal parts of the shower aid can cool down considerably, so avoid contact with the skin.
- The seat and upholstery materials meet the requirements for resistance to flammability in accordance with EN 1021-1 / ISO 8191-2, but do not expose upholstery or other parts to sources

of fire, such as cigarettes.

- Reporting of incidents:

If you as the operator, user or their relatives notice possible defects or functional limitations, please contact an authorised specialist dealer immediately. Operator, user or their relatives should inform your specialised dealer who provided you with the product about suspected serious incidents that have had or could have directly or indirectly one of the following consequences and that affect them. You can also report this to the competent higher federal authority.

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration in the state of health of a patient, user or other person,
- a serious threat to public health.

2.1. Indication

The use of the shower aid for adults is indicated in the event of unsteadiness or inability to stand due to functional impairment:

- of the lower extremities with restricted or cancelled mobility
- muscle strength (e.g. loss of strength due to malignant diseases, chronic, advanced cardiovascular and lung diseases) and/or
- damage to the vestibular functions or tendency to fall due to other causes with a sufficiently stable trunk posture for sitting and sufficient residual functions of the upper extremities.

2.2. Contraindication

This shower aid for adults is unsuitable for

- Inability to sit
- Severe limitations of cognitive ability
- Perceptual disorders
- Balance disorders

2.3. Declaration of Conformity

We Bischoff & Bischoff GmbH declare under our sole responsibility that the DH-40 and DH-40 L shower aid complies with all applicable requirements of MDR 2017/745.

2.4. Responsibility

We can only provide a warranty if the product is used under the specified conditions and for the intended purposes, if modifications, extensions, repairs and maintenance work are only carried out by persons authorised by us and if the shower aid is used in compliance with all instructions for use. A warranty is also excluded in the event of damage caused by improper handling of fire sources, moisture and liquids such as hair dryers, cigarettes, cleaning agents or chemicals.

3. Intended use

The use of the shower aid increases independence when using the shower. The maximum user weight of 110 kg must not be exceeded. The chair must not be used as a climbing aid or for similar purposes. The recommended body height is 1.6 m to 2.0 m.

4. Product and delivery overview

4.1. Inspection of the delivery

All Bischoff & Bischoff products undergo a proper final inspection in our company and are labelled with the CE mark.

The Bischoff & Bischoff shower aid is supplied in a special cardboard box. Delivery and instruction of the product is carried out by a qualified medical supply retailer.

After unpacking, keep the cardboard box if possible. It can be used to store or return the product at a later date.

In the case of dispatch by rail or forwarding agent, the goods must be checked immediately for transport damage in the presence of the bearer.

If there is any damage, please contact our customer service immediately.

Check that the contents are complete (see section 4.2) and undamaged. In the event of irregularities or damage, please contact our customer service.

4.2. Scope of delivery


Your new Bischoff & Bischoff shower aid is delivered in a Bischoff & Bischoff original box. To prevent damage during transport, attachments are supplied separately. The scope of delivery in the special box includes

- Shower aid
- Operating instruction


4.3. Overview

To make the handling of your Bischoff & Bischoff shower aid as easy as possible, we have provided the written instructions. The following illustrations are intended to help you with the designation and interaction of the individual assemblies. The following illustrations are intended to help you with the designation and interaction of the individual assemblies (see Fig. 1).

4.3.1. Shower aid with backrest

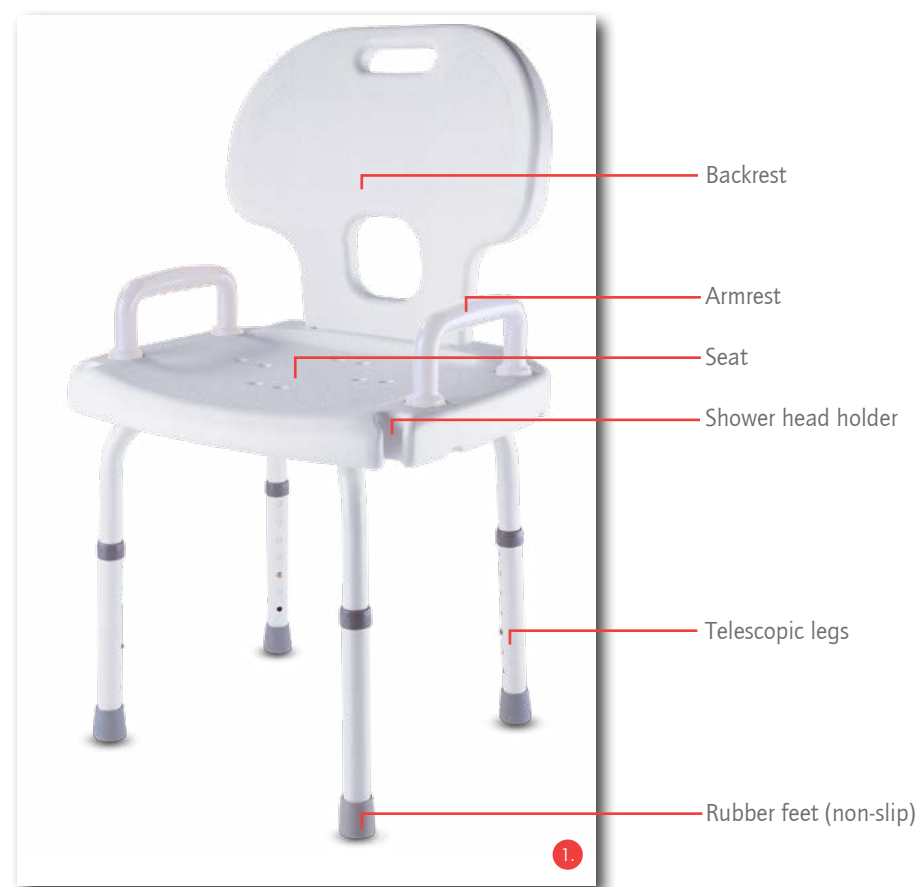
4.3.2. Shower aid without backrest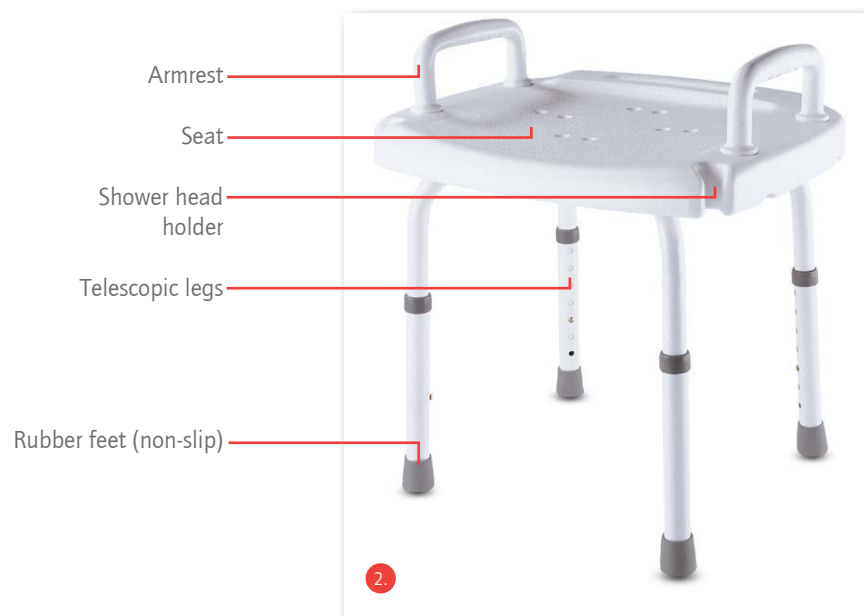

4.4. Type plate and serial number

The type plate and serial number are located on a frame tube.

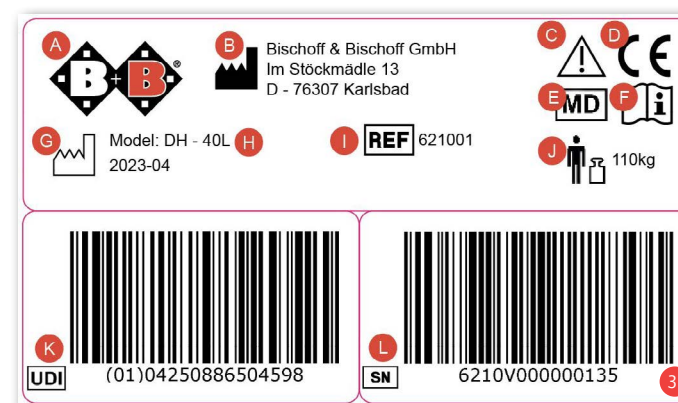- A** Manufacturer logo
- B** Manufacturer
- C** Information Attention! - Observe instructions for use
- D** CE mark
- E** Medical device
- F** Important! - Observe instructions for use
- G** Production date
- H** Model designation
- I** Model number
- J** Max. user weight
- K** UDI Part number
- L** Serial number

5. Assembly and customisation

You can adjust your shower aid yourself with a little manual dexterity. If you have any questions, please contact your specialist dealer.

5.1. Assembly

The shower aid is supplied in individual parts. Assembly is very simple and can be carried out in accordance according to the following instructions. All you need is a Phillips screwdriver. If you have any questions about fitting, please contact the medical supply retailer who supplied you with the product

Caution: risk of jamming! When assembling there is a risk of entrapment.

5.2. Mounting the legs

- Place the seat surface with the underside facing upwards on a stable ground and first screw the legs with the labelling (1) to the two outer holes (Fig. 4)
- Then screw the legs with the labelling (2) onto the seat and tighten all screws firmly (Fig. 5).

Attention! Observe the correct sequence. Only This ensures correct fastening.

4.2. Mounting the handles

- Lay the shower aid on its side and insert the handles into the openings provided (Fig. 6).
- Tighten the plastic screws underneath the seat by hand (Fig. 7).



5.3. Mounting the backrest

- Stand from behind the safely positioned shower aid.
- Take the backrest and press it into the recess. Pay attention to the alignment. The backrest only fits into the recess in one direction (Fig. 8).

Caution! The lower part of the backrest must protrude approx. 5 mm from the underside of the seat.

5.4. Adjusting the seat height

- Place the shower aid on its side to take the strain off your legs.
- Press in the push button and move the outer tube.
- Slide the outer tube to the desired seat height and engage the push button in the corresponding hole (Fig. 9).
- Repeat the adjustment for the other three legs, ensuring the same height setting.

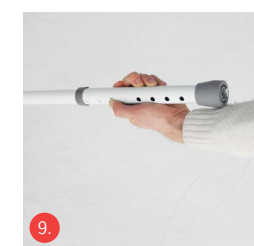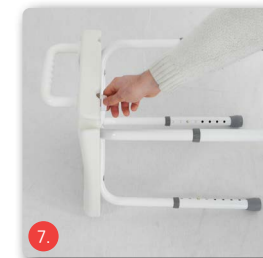Caution! The press studs must visibly engage in the holes.

Important! The handles and backrest can only be fitted by persons with sufficient strength due to the high operating forces.

6. Check before use

Before use, ensure that all screws are tight and that the chair is standing on a level surface.

After adjusting the height of the shower aid, it is particularly important to ensure that the spring pins are correctly engaged. The rubber feet are a wearing part and must not be damaged. If they are damaged, they can be replaced by your specialist dealer if necessary.

6.1. Using the shower aid

Place the shower aid in the shower. Make sure that all four rubber feet are securely on the floor or shower tray. People who are unsure about using the shower should only enter it with the help of a carer.

Use the shower aid like a normal aid.

Do not lean backwards or too far to the side, otherwise there is a risk of tipping over.

The upholstery material is made of water-repellent plastic.

Use the handles to sit down and stand up.

You can attach the shower head to the front of the seat (right and left) during personal hygiene (Fig. 10).

Now you can get on with your personal hygiene. Excess water drains away quickly through the holes in the seat.

Remove any soap residue after use and dry the shower aid.

Caution! Shower aids are generally less resistant to tipping than chairs in the home. Therefore, avoid leaning out to the side.

Lifting points

You can lift the shower aid by the metal frame to carry it over obstacles or steps. Avoid lifting the aid by the seat.

Beware of the risk of slipping! There is a risk of slipping when sitting down or standing up in wet rooms, especially if there is soap on the floor.

7. Technical data

Dimensions and weights

	DH-40	DH-40 L
Max. Load capacity (kg):	110	110
Width (mm):	500	500
Depth (mm):	410	410
Height (mm):	500 - 680	785 - 965 (incl. handle / back)
Seat width (mm):	380	380
Seat depth (mm):	320	320
Seat height (mm):	390 - 570	390 - 570 (in 7 levels)
Empty weight (kg):	3,2	4,1

Materials

Seat and backrest:	Plastic (PSE), white
Fixing screws:	Stainless steel
Legs:	Aluminium, lacquered
Feet:	Rubber

The specified dimensions are subject to manufacturing tolerances of +/- 10 mm

8. Care instructions for the user

8.1. Cleaning

The seat and backrest materials can be cleaned with warm water with the aid of a sponge and/or a soft brush. For stubborn dirt, a commercially available cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents, such as solvents / abrasive cleaners or hard brushes.

Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information for the cleaning agent.

- Plastic panelling is attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the shower aid with a damp cloth.
- For heavier soiling, also use a mild cleaning agent.
- Wash the armrests, seat and backrest upholstery with a mild detergent.
- Upholstery parts can be washed at 40°C. Dry at room temperature.

8.2. Disinfection

The shower aid must be disinfected after use. The shower aid can be disinfected with a standard household disinfectant. Use approved spray disinfectants or alcohol-based wipe disinfectants.

- In the case of multiple product users, a commercially available disinfectant is prescribed.
- Only use wipe disinfectants approved by the Robert Koch Institute (RKI) and the Verbund für Angewandte Hygiene e.V. (e.g. Bacillol AF or Kohrsolin FF); see manufacturer's recommendations for contact time and concentration.
- Caution: Damaged upholstery parts cannot be adequately disinfected. These should be replaced.
- Disinfectants can affect the surfaces under certain circumstances. This can result in longer-term functional restrictions. Please refer to our enclosed overview of the most important care, hygiene and maintenance work.

Caution: Risk of infection! The shower aid must be cleaned after each use.

Caution! Do not use a high-pressure or steam cleaner.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialist dealer will be happy to provide you with these.

9. Maintenance

9.1. Maintenance instructions

Your shower aid is made of high-quality materials for long and comfortable use. Each product is thoroughly before it leaves our premises. Nevertheless, it is possible that a defect may occur. If you notice a fault with your shower aid, it must no longer be used. Please contact the medical supply retailer who supplied you with the product for repair. They will be happy to help you. Before using your shower aid, please check regularly that the rubber feet, armrests and backrest, the upholstery and the height adjustment are not damaged and are fully functional.

You must check your shower aid regularly according to the maintenance schedule and ensure that it is in perfect condition.

9.2. Maintenance instructions for the specialised trade

Typical wearing parts are the rubber feet. Contact your specialist dealer for spare parts. We recommend that you bring your shower aid to your specialist dealer for inspection at least once a year.

In the event of faults or defects, return the shower aid to your specialist dealer immediately for repair.

The specialist dealer will be provided with the necessary information and documents for repair and maintenance on request.

Please contact Bischoff & Bischoff customer service if you have any questions. You can find the customer service contact details on the back of these operating instructions or on our website www.bischoffbischoff.com.

Check your shower aid regularly according to the maintenance schedule.

10. Repair

Repairs and maintenance work may only be carried out by authorised partners (medical supply stores or specialist dealers). Please call our customer service to find a partner in your area.

11. Storage and dispatch

If the shower aid is to be stored or shipped, all insertable and unattached parts must be removed and packed individually. The individually packed parts can then be packed together in a larger box. It is advisable to keep the original packaging and store it for these purposes so that it is available when needed. Your shower aid will then be optimally protected against environmental influences during storage or transport.

When storing your shower aid, please observe the information in the technical data. To prevent mould growth and damage to the upholstered parts, the storage location should be as dry as possible and not exposed to direct sunlight.

12. Waste disposal

If your shower aid is no longer in use and is to be disposed of, please contact your specialist dealer.

If you would like to dispose of the waste yourself, enquire with local recycling companies about the regulations in your area.

13. Passing on reuse

The shower aid is suitable for reuse. Before it is passed on to another user, it must be serviced and sanitised by a specialist dealer. When passing on the shower aid, please remember to hand over these operating instructions and the address of your specialist dealer to the new user.

The shower aid must be checked according to the maintenance schedule and be in perfect working order.

14. Useful life

The expected service life of the DH-40 and DH-40 L is up to 5 years if used as intended and the safety, care and maintenance instructions are followed. The shower aid can continue to be used beyond this period if it is in a safe condition.

15. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.
2. This warranty excludes claims, if a repair or replacement of a product or a part is required for the following reasons:
 - a) Normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
 - b) Overloading the product such as, for example, by exceeding the maximum user weight or load.
 - c) The product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
 - d) Accessories have been used which are not original accessories.
 - e) The product or part has been damaged through negligence, accident or incorrect use.
 - f) Changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
 - g) Repairs have been carried out, before our customer service has been informed of the circumstances.
3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.
4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).
5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.
6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff & Bischoff. Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will be happy to provide these to you.
For replacement devices and repairs, the warranty period is one year.

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

Guarantee certificate

Product: DH-40 / DH-40 L

Serial number:*

* (to be entered by the dealer)

Specialist dealer:

Date and stamp

(Please note our general terms and conditions)

*Information signs are located on the side frame

Please note!

Despite careful research and editing, incorrect information may have crept into our operating instructions. We reserve the right to make technical changes. Illustrations may contain accessories. The latest version of the operating instructions can be found in the download area on our website: **www.bischoff-bischoff.com**

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