# Sofort- Kältekompresse

Größe: 15 x 17 cm

## Instant Cold Pack

size: 15 x 17 cm

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**Intended use:**

Cold pack is used for providing cold therapy for temporarily relief of pain, reduces swelling, and aids in cooling. Suitable for local cold therapy in medical institutions or at home. Refer to the warning instructions for restrictions.

**Indications:**

- Relieves local pain and swelling caused by sprains, strains, impacts, bruises, and other injuries. Within 48–72 hours after the injury.
- Local pain and discomfort caused by toothache and mosquito bites.
- Physical cooling for patients with high fever
- Local pain and swelling caused by mild burns
- Used to reduce swelling, pain, and bleeding around the wound after surgery.

**Contraindications:**

- Raynauds Phenomenon
- Cold Hypersensitivity
- Cardiac Conditions
- High Blood Pressure
- Superficial Nerves
- Healing Wounds

**Use method:**

1. Shake contents to the bottom of the bag.  
Locate the inner pouch and squeeze firmly to rupture inner fluid pouch.
2. Shake briefly to mix contents.
3. Wrap the cold pack in a towel or clothes.
4. Apply to the affected area, the recommended cooling time is 15–20 minutes.

**Undesirable side-effects:**

Avoid pressing the cold pack directly on the skin, as this may cause cold injury. In most cases, the symptoms are transient and typically resolve within a few hours. If the symptoms do not alleviate and exceed one day, please consult a physician.

**Intended clinical benefits:**

Relieve pain, control inflammation and swelling, promote hemostasis, physically cool down, and control postoperative swelling.

**Intended user:**

The cold pack shall only be used by adults who can read and understand the instruction manual.

**Intended patient population:**

The cold pack is applied to a wide range of people who require cold therapy.

The cold pack shall not be applied to following patients:

- Sensory impaired or unconscious patients
- Pregnant women
- Patients situations mentioned in the contraindications
- Do not use on infants

**Temperature performance:**

Meets requirements at room temperature: 20–24°C

- For specifications of 15 x 23 cm and above, the minimum cooling temperature shall not be higher than 4 °C, and the temperature rise shall not be higher than 10 °C after 15 minutes.
- For specifications of 15 x 23 cm and below, the minimum cooling temperature shall not be higher than 4 °C and shall not be higher than 12 °C after 15 minutes.

**Technical data:**

Storage/Transport conditions: At a temperature between 4°C and 35°C, avoid direct sunlight

Main contents: Urea, Wasser

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Diemnsions: 15 x 17 cm

**Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.**

**Warnings:****⚠ Attention!**

1. For external use only.
2. Consult instructions for use.
3. Check that the packaging bag is without leakage and the product is within its expiry date before use.
4. Do not use on infants.
5. Harmful if swallowed. If accidentally swallowed, contact a poison control center or physicians immediately.
6. If the bag breaks and fluid comes in contact with skin or eyes, flush with water immediately and call a physician.
7. Individuals with circulatory problems should consult a physician before using this product.
8. Pay attention when used by the elderly, children and people with communication barriers.
9. Prior to use, store at room temperature.
10. Always wrap a towel or cloth around the cold pack before applying to bare skin.
11. If the cold pack feels too cold or if there are significant skin color changes, more layers of toweling should be added or the cold pack should be removed.
12. Patients should not lie with their full body weight on top of a cold pack.
13. Do not open the cold pack.
14. Do not reuse by freezing or refrigeration.
15. Keep out of reach of children.

**Disposal:**

Dispose of contents/container in accordance with local/regional/national/international regulations.

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 **Hersteller**  
Manufacturer

 **nextRESCUE**  
Brennhorster Str. 12  
32479 Hille  
Germany

 **JIANGSU INTCO MEDICAL PRODUCTS CO., LTD.**  
No. 298 Yandunshan Road, Zhenjiang New District  
212132 Jiangsu, P.R. China

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 **MedNet EC-REP C llb GmbH**  
Borkstrasse 10  
48163 Muenster  
Germany


**Manufacturer reference | Hersteller-Referenz:**  
C150170NC


**1.** Shake contents to the bottom of the bag.  
Locate the inner pouch and squeeze firmly to rupture inner fluid pouch.

**2.** Shake briefly to mix contents.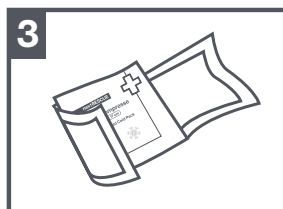

**3.** Wrap the cold pack in a towel or clothes.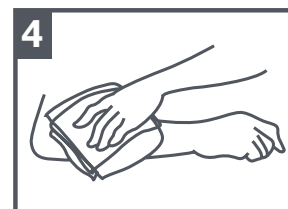

**4.** Apply to the affected area, the recommended cooling time is 15–20 minutes.