
SAM ThoraSite®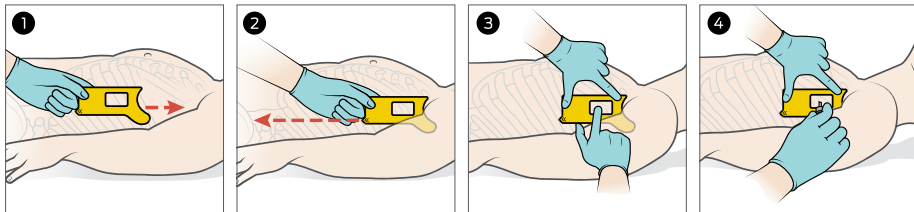

EN INSTRUCTIONS FOR USE

sammedical.com/eIFU

1. Push Axilla Hook into the armpit until fully inserted into axillary fossa.

Note: Keep arm positioned <90° from patient's side.

2. Point Alignment Arrow toward the hip, aligning with anterior iliac crest.

Note: Base of device should be parallel to patient's spine.

3. Palpate within Safe Zone Window to feel intercostal space (ICS).

Note: If two intercostal spaces are identified within the Safe Zone Window, select the upper ICS.

4. Vent/Decompress in identified ICS within the Safe Zone Window.

Note: If ribs cannot be palpated, vent/decompress in the center of the Safe Zone Window.

If rib is encountered, redirect above the rib.

INTENDED USE/PURPOSE: An anatomical location aid to identify an appropriate location for lateral thoracostomies in adult and adolescent patients. Not intended for pediatric, or neonatal use.

INDICATIONS FOR USE: Indicated for use by trained professionals following their standard of care to treat patients with pneumothorax, hemothorax and similar conditions requiring a thoracostomy.

Notice to Users/Patients: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

CAUTION:

REUSE OF DEVICE MAY CAUSE ILLNESS OR INJURY. DISPOSE OF USED DEVICE PER HEALTHCARE FACILITY BIOHAZARD PROTOCOL.

SYMBOL GLOSSARY:

Manufacturer	
Lot/Batch Code	LOT
Catalog Number	REF
Do Not Reuse	
Caution	
Medical Device	MD
Authorized representative in the European Union	EU REP
Authorized representative in the Swiss Confederation	CH REP
Authorized representative in the United Kingdom	UK REP
CE Mark	CE
Importer	
Consult Instructions for Use	
Unique Device Identifier	UDI

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All rights reserved.
Emergo Consulting (UK) Limited